




**Nucleus® Freedom™
Bodyworn Sound
Processor
Quick Reference Guide**



Nucleus® Freedom™ Bodyworn Speech Processor Quick Reference Guide

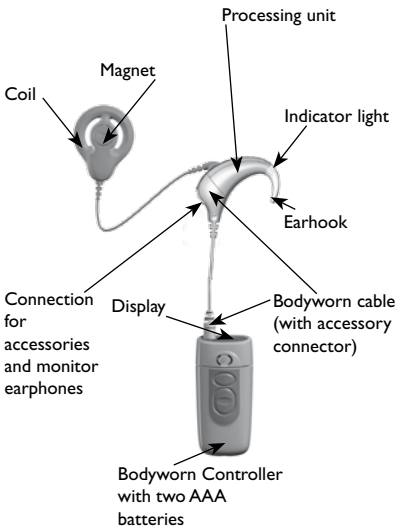

Parts of your speech processor

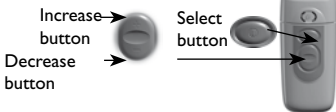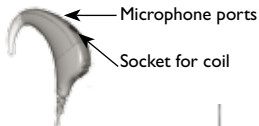

On and off

Press the Select button until you hear sound or see the display.



Changing the program

Press the Select button repeatedly until you reach the program you want.



Changing microphone sensitivity

Press the Increase or Decrease buttons until you reach the level you want.



Changing batteries



1. Slide the cover down.




2. Change the batteries, making sure the + and - symbols on the batteries match the + and - symbols in the compartment.



3. Slide the cover back until it clips into place.


Using an accessory and/or the monitor earphones

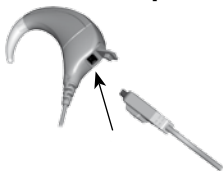

Plug in the accessory directly. If you want to check the sound of the processor, plug in the monitor earphones. To check the sound of an accessory, plug the accessory into the monitor earphones.

To activate the accessory or monitor earphones, push the Increase and Decrease buttons at the same time for at least one second.

To select sound from the accessory, the in-built telecoil, or the microphone in turn, push the Increase and Decrease buttons at the same time for at least one second.

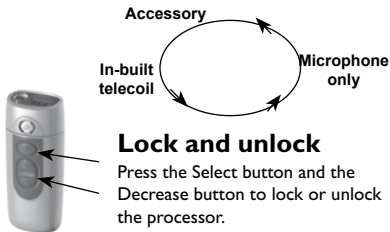

Lock and unlock

Press the Select button and the Decrease button to lock or unlock the processor.

Changing volume



1. Briefly push *either* the Increase or Decrease button for more than one second, until you hear a beep and/or see a flashing 'V' on the display.
2. Continue briefly pushing either the Increase or Decrease buttons until you reach the volume level you want.

Using the in-built telecoil

Push both the Increase and Decrease buttons until 'T' shows on the display.



Resetting the speech processor



Press all three buttons (Select, Increase and Decrease) together (for at least one second) to return to the default settings.

Mic lock

Attach the Mic lock as shown.



Changing the microphone protector



Slide your fingernail along the base to remove the cover. Replace microphone protector so it fits securely.






Put cover back on. Check the cover every week and replace if dirty.

Display Key

M	Microphone
T	Telecoil
MT	Microphone and Telecoil on at the same time
S	Sensitivity (ranges from 0 to 20)
V	Volume (ranges from 0 to 9)
+L	Button controls are locked and cannot be adjusted
-L	Button controls are unlocked and can be adjusted
EA	External Accessory: flashes when accessory on
≡	Lines on left: processor programmed for left ear
≡	Lines on right: processor programmed for right ear

Specific symbols for Bodyworn

- X External Accessory connected
-  Sound Indicator: The processor is picking up sounds
-  Battery Indicator: Battery power full
-  One segment of Battery Indicator: Battery power low



Help message	Meaning	Action
H1	Flat battery	Change all batteries. Note: you will stop hearing sound and only be able to turn off the speech processor.
H2	Low battery	Change all batteries.
H3	Coil error	Check: <ul style="list-style-type: none"> • coil position • cable connections • for cable damage.
H4	Audio error	Follow the steps in the Troubleshooting section of your User Manual to try to resolve the problem.
H5	Program corrupted	Use an alternate program when it is available until you are able to return the speech processor to your clinician. They can install a new program for you.

Water

Your Freedom speech processor is protected against splashing water. Take care not to immerse it in water.

If accidentally immersed in water, wipe your processor dry, remove the Controller, microphone protector and coil from the Processing Unit and remove the batteries from the Controller. Place all parts in your dry aid kit for at least twelve hours.

If it then does not work, return it to your clinician. Cochlear cannot guarantee that they will be able to repair any water-damaged part.

Program	Description of the program and/or when to use the program
P1	
P2	
P3	
P4	

Magnet strength:

Transmitting cable length:

Cochlear Ltd, Sydney- Australia, Tel: 61 2 9428 6555 Fax: 61 2 9428 6353
Cochlear AG, Basel- Switzerland, Tel: 41 61 205 04 04 Fax: 41 61 205 0405
Cochlear Europe Ltd, Surrey- UK, Tel: 44 1932 87 1500 Fax: 44 1932 87 1526
Cochlear GmbH, Hannover- Germany, Tel: 49 511 542 770 Fax: 49 511 542 7770
Cochlear Benelux NV, Mechelen- Belgium, Tel: 32 15 36 2877 Fax: 32 15 36 2870
Cochlear France S.A.S., Saint Aubin- France, Tel: 33 811 111 993 Fax: 33 160 196 499
Cochlear Italia SRL, Bologna- Italy, Tel: 39 051 34 3578 Fax: 39 051 39 2062
Cochlear Nordic AB, Mölnlycke- Sweden, Tel: 46 31 335 1461 Fax: 46 31 335 1460



Cochlear™ www.cochlear.com

Nucleus is a registered trademark of Cochlear Limited.
Cochlear, the elliptical logo and Freedom are trademarks of
Cochlear Limited. © Cochlear Limited 2006

Printed in Switzerland
N30863F Iss 2