
Changing Coil Cables


TIPS FOR NUCLEUS® SOUND PROCESSORS

Changing coil cables is something you should only have to do occasionally, if you are fitting a microphone cover sleeve or if you are replacing a coil cable. Remember when storing your processor overnight in the Dry & Store unit, you should leave the coil connected and the microphone cover sleeve on (if fitted).


Using the correct method for changing coil cables reduces the risk of accidentally damaging the coil cable and the coil cable sockets in the processing unit and the coil.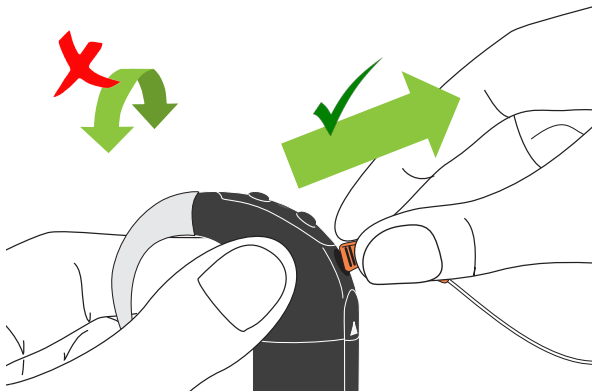


1. Hold the coil cable by the grips and firmly pull it straight out of the socket on the processing unit. Do not twist.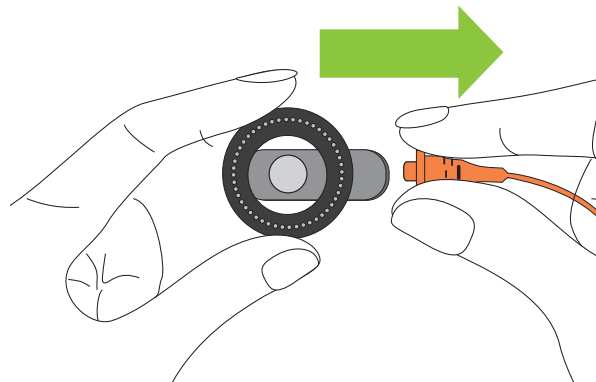

2. (If your coil and cable are a one-piece unit, go to step 4.) Hold the base of the plug and firmly pull the coil cable straight out of the coil.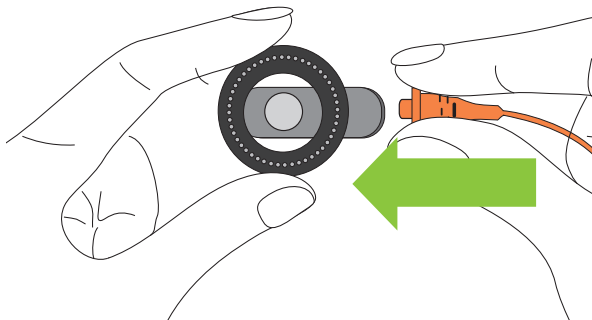

3. (If your coil and cable are a one-piece unit, go to step 4.) To attach the new coil cable, push the coil cable plug into the coil until it clicks into place.

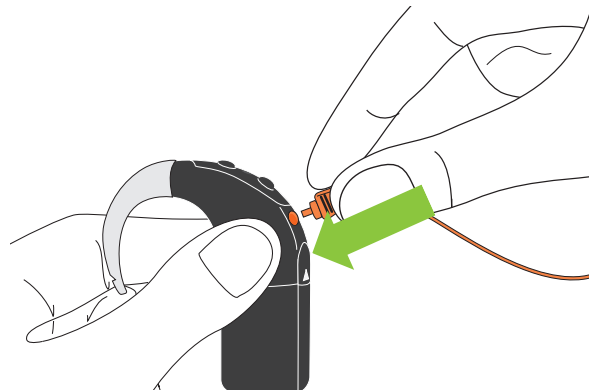4. Push the coil cable into the coil cable socket on the processing unit until it clicks into place.

To avoid breaking the coil cable, do not pull on the coil cable and do not twist the coil cable plug.

For more information, see your sound processor's User Guide.

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© Cochlear Limited 2014 588807 ISS2 AUG15