





Nucleus® Freedom™ Bodyworn Speech Processor Quick Reference Guide

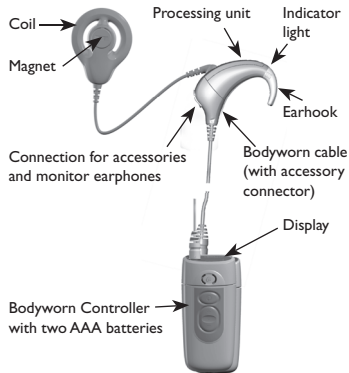

Parts of your speech processor

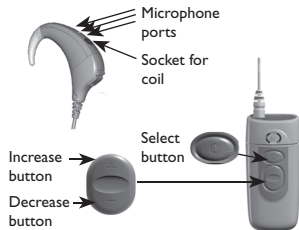

On and off

Press the Select button until you hear sound or see the display.

Changing the program


Press the Select button repeatedly until you reach the program you want.

Changing microphone sensitivity

Press the Increase or Decrease buttons until you reach the level you want.



Changing batteries

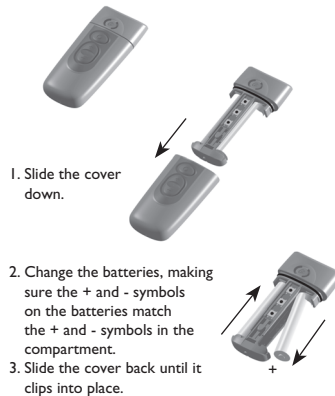

Water

Your Freedom speech processor is protected against splashing water. Take care not to immerse it in water. If accidentally immersed in water, wipe your processor dry, remove the Controller, microphone protector and coil from the Processing Unit and remove the batteries from the Controller. Place all parts in your dry aid kit for at least twelve hours. If it then does not work, return it to your clinician. Cochlear cannot guarantee that they will be able to repair any water-damaged part.

Program	Description of the program and/or when to use the program
P1	
P2	
P3	
P4	

Magnet strength:
Transmitting cable length:

Cochlear Ltd (ABN 96 002 618 073) 14 Mars Road, Lane Cove NSW 2066, Australia
Tel: 61 2 9428 6555
Cochlear Americas USA Tel: 1 303 790 9010
Cochlear AG Switzerland Tel: 41 61 205 0404
Cochlear Deutschland GmbH & Co. KG Germany Tel: 49 511 542 770
Nucleus is a registered trademark of Cochlear Limited.
Cochlear, the elliptical logo and Freedom are trademarks of Cochlear Limited.
© Cochlear Limited 2006

Printed in Australia
N32079 ISS2


Using an accessory and/or the monitor earphones



Plug in the accessory directly. To check the sound of the processor, plug in the monitor earphones. To check the sound of an accessory, plug the accessory into the monitor earphones.

To activate the accessory or monitor earphones, push the Increase and Decrease buttons at the same time for at least one second.

To select sound from the accessory, the in-built telecoil, or the microphone in turn, push the Increase and Decrease buttons at the same time for at least one second.

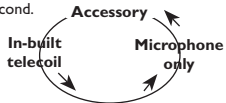

Lock and unlock

Press the Select button and the Decrease button to lock or unlock the processor.


Changing volume



1. Briefly push *either* the Increase or Decrease button for more than one second, until you hear a beep and/or see a flashing 'V' on the display.
2. Continue briefly pushing either the Increase or Decrease buttons until you reach the volume level you want.

Using the in-built telecoil

Push both the Increase and Decrease buttons until 'T' shows on the display.

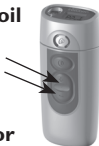

Resetting the speech processor



Press all three buttons (Select, Increase and Decrease) together (for at least one second) to return to the default settings for a MAP.

Mic lock

Attach the Mic lock as shown.



Changing the microphone protector



Slide your fingernail along the base to remove the cover. Replace microphone protector so it fits securely.



Put cover back on. Check the cover every week and replace if dirty.

Display Key

- M Microphone
- T Telecoil
- MT Microphone and Telecoil on at the same time
- S Sensitivity (ranges from 0 to 20)
- V Volume (ranges from 0 to 9)
- +L Button controls are locked and cannot be adjusted
- L Button controls are unlocked and can be adjusted
- EA External Accessory: flashes when accessory on
- ≡ Lines on left: processor programmed for left ear
- ≡ Lines on right: processor programmed for right ear

Specific symbols for Bodyworn

- X External Accessory connected
- ||||| Sound Indicator: The processor is picking up sounds
- ☑ Battery Indicator: Battery power full
- ▢ One segment of Battery Indicator: Battery power low



Help message	Meaning	Action
H1	Flat battery	Change all batteries. Note: you will stop hearing sound and only be able to turn off the speech processor.
H2	Low battery	Change all batteries.
H3	Coil error	Check: • coil position • cable connections • for cable damage.
H4	Audio error	Follow the steps in the <i>Troubleshooting</i> section of your User Manual to try to resolve the problem.
H5	Program corrupted	Use an alternate program when it is available until you are able to return the speech processor to your clinician. They can install a new program for you.