

# Cochlear™

## Quick Guide to the Nucleus® CR210 Remote Control

- ☒ Getting started
- ☒ Quick reference



*Hear now. And always*



# About

Use the Cochlear™ Nucleus® CR210 Remote Control to control the commonly used functions of your Cochlear Nucleus CP910, CP920 or Kanso (CP950) Sound Processor.\*

\* Not all products are approved in all countries.

## NOTE

More information and warnings in the *CR210 Remote Control User Guide*.



# CR210 Remote Control



ABOUT



# Change battery

Change the battery whenever you see the low battery indicator.



**When display shows low battery in remote control:**



Unscrew cover with a Phillips head (cross-head) screwdriver.



Remove cover and old battery.



## CAUTION

Use **ONLY** disposable CR2032 or 5004LC 3V standard lithium coin cell batteries in your remote control.



Carefully insert new battery, plus side "+" up.



Replace cover, bottom edge first.



Screw cover back on.



POWER

## Turn on

Slide the switch  
up to turn on.



# Turn off

Slide the switch  
down to turn off.



TURN ON

# Pair

Pairing allows two devices to communicate with each other.



**When display shows remote control is not paired:**



Turn on both your processor and remote control.

Place coil/processor on the back of your remote control.





Remote control shows pairing in progress.



Remote control shows it's paired.



If you have a second processor, repeat steps 2–4.



TURN ON

# Change programs



# Adjust volume/sensitivity



Turn up either  
volume or  
sensitivity  
(as set up by  
your clinician)

Turn down  
volume or  
sensitivity



USE

# Wireless Accessories

When a Cochlear Wireless Accessory (Mini Microphone or TV Streamer) is paired with your processor, use your remote assistant to start and stop streaming audio.

Display shows  
Wireless  
Accessory:



Turn Wireless  
Accessory on:



Long-press once.

Turn Wireless  
Accessory off:



Long-press once.

# Plug-in audio accessories

When a plug-in audio accessory is connected to your processor, use your remote control to turn the accessory off and back on.

Display shows  
plug-in audio  
accessory:



Turn plug-in  
audio accessory  
off:



Short-press once.

Turn plug-in  
audio accessory  
on:



Short-press once.



USE

# Telecoil

Listen to hearing loops and hear voices on the telephone with telecoil or auto telecoil (if available). Auto telecoil detects the telecoil signal automatically.

Turn telecoil on:



Short-press once.

Turn telecoil  
to auto:



Short-press twice.

Turn telecoil off:



Short-press once.

# Care

Clean your remote control with a soft dry cloth.  
Keep your remote control dry.



CARE



**Cochlear Ltd** (ABN 96 002 618 073)

1 University Avenue, Macquarie University, NSW 2109, Australia

Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

**Cochlear Ltd** (ABN 96 002 618 073)

14 Mars Road, Lane Cove, NSW 2066, Australia

Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

**Cochlear Americas**

13059 E Peakview Avenue, Centennial, CO 80111, USA

Tel: +1 303 790 9010 Fax: +1 303 792 9025

**ECREP Cochlear Deutschland GmbH & Co. KG**

Karl-Wiechert-Allee 76A, 30625 Hannover, Germany

Tel: +49 511 542 770 Fax: +49 511 542 7770

**株式会社日本コクレア (Nihon Cochlear Co Ltd)**

〒113-0033 東京都文京区本郷2-3-7 お茶の水元町ビル

Tel: +81 3 3817 0241 Fax: +81 3 3817 0245

**www.cochlear.com**

Cochlear, Hear now. And always, Nucleus, and the elliptical logo  
are either trademarks or registered trademarks of Cochlear Limited.

© Cochlear Limited 2016

*Hear now. And always*

402610 ISS8 APR16