

# CR310 Remote Control

Quick Guide

- ✓ Getting started
- ✓ Quick reference



*Hear now. And always*



# About

Use the Cochlear™ CR310 Remote Control to control the commonly used functions of your Cochlear Nucleus 7 Sound Processor (model number: CP1000).



## **NOTE**

More information and warnings in the *CR310 Remote Control User Guide*.



## **WARNING**

Swallowing Lithium batteries may lead to serious injury or death. If swallowed, immediately go to a hospital emergency room. Keep out of reach of children. Keep in original packaging until ready to use. Dispose of used batteries immediately.

# CR310 Remote Control



# Change battery

Change the battery whenever you see the low battery indicator.



1. Unscrew cover with a Phillips head (cross-head) screwdriver.




2. Remove cover and old battery.



## CAUTION

Use **ONLY** disposable CR2032 or 5004LC 3V standard lithium coin cell batteries in your remote control.

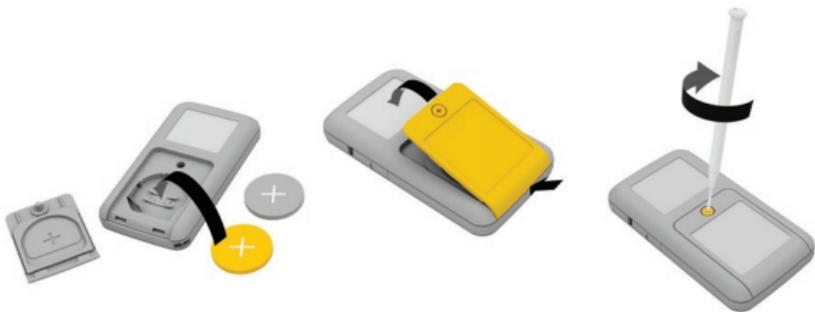

3. Carefully insert new battery, plus "+" side up.
4. Replace cover, bottom edge first.
5. Screw cover back on.

# Turn on

Slide the switch up to turn on.



# Turn off

Slide the switch  
down to turn off.



# Pair

Pairing allows two devices to communicate with each other.



**When display shows remote control is not paired:**

1. Turn your processor and remote control **off**, then **on**.
2. Place the coil on the back of your remote control.



3. Remote control shows pairing in progress.



4. Remote control shows it's paired.




5. If you have a second processor, repeat steps 1–4.

# Change programs

Next program

Previous program

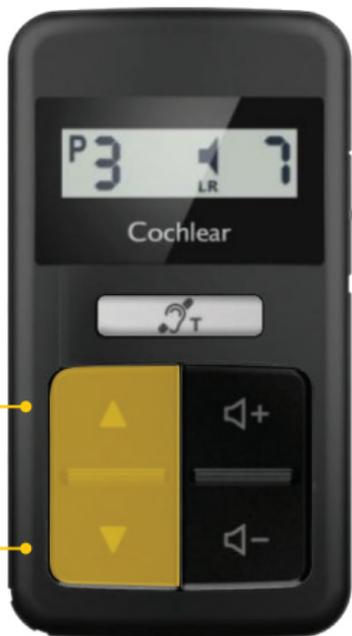

# Adjust volume/sensitivity



Turn up either  
volume or  
sensitivity  
(if available)

Turn down  
volume or  
sensitivity

## Wireless accessories

When a Cochlear True Wireless™ Accessory (Mini Microphone or TV Streamer) is paired with your processor, use your remote control to start and stop streaming audio.

Display shows wireless accessory:



Start streaming:



Long-press once.

Stop streaming:



Short-press once.

## FM accessory

When an FM accessory is connected to your processor, use your remote control to turn the accessory off and back on.

Display shows  
FM accessory:



Turn FM  
accessory off:



Short-press once.

Turn FM  
accessory on:



Short-press once.

# Telecoil

Listen to room loops with telecoil.

Turn telecoil on:



Short-press once.

Turn telecoil off:



Short-press once.

## Care

Clean your remote control with a soft dry cloth.  
Keep your remote control dry.



Hear now. And always



**Cochlear Ltd** (ABN 96 002 618 073)

1 University Avenue, Macquarie University, NSW 2109, Australia

Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

**Cochlear Ltd** (ABN 96 002 618 073)

14 Mars Road, Lane Cove, NSW 2066, Australia

Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

 **Cochlear Deutschland GmbH & Co. KG**

Karl-Wiechert-Allee 76A, 30625 Hannover, Germany

Tel: +49 511 542 770 Fax: +49 511 542 7770

**Cochlear Americas**

13059 E Peakview Avenue, Centennial, CO 80111, USA

Tel: +1 303 790 9010 Fax: +1 303 792 9025

**株式会社日本コクレア (Nihon Cochlear Co Ltd)**

〒113-0033 東京都文京区本郷2-3-7 お茶の水元町ビル

Tel: +81 3 3817 0241 Fax: +81 3 3817 0245

**[www.cochlear.com](http://www.cochlear.com)**

Cochlear, Hear now. And always, Nucleus, and the elliptical logo are either trademarks or registered trademarks of Cochlear Limited.

© Cochlear Limited 2017

594632 ISS2 APR17