

CR210 Remote Control for Kanso[®]

Quick Guide



Getting started



Quick reference

CP950

Hear now. And always



Cochlear[®]

About

Use the Cochlear™ Nucleus® CR210 Remote Control to control the commonly used functions of your Cochlear Kanso® Sound Processor (model number: CP950).

**NOTE**

More information and warnings for the CR210 Remote Control in its user guide.

CR210 Remote Control



Change battery

Change the battery whenever you see the low battery indicator.



1. Unscrew cover with a Phillips head (cross-head) screwdriver.



2. Remove cover and old battery.



CAUTION

Use **ONLY** disposable CR2032 or 5004LC 3V standard lithium coin cell batteries in your remote control.



3. Carefully insert new battery, plus side “+” up.
4. Replace cover, bottom edge first.
5. Screw cover back on.

Turn on

Slide the switch up to turn on.



Turn off

Slide the switch
down to turn off.



TURN ON

Pair

Pairing allows two devices to communicate with each other.



When display shows remote control is not paired:

1. Turn on both your processor and remote control.
2. Place the processor on the back of your remote control.



3. Remote control shows pairing in progress.



4. Remote control shows it's paired.



5. If you have a second processor, repeat steps 2–4.

Change programs

Next program

Previous program



Adjust volume/sensitivity



Turn up either
volume or
sensitivity
(if available)

Turn down
volume or
sensitivity

Wireless accessories

When a Cochlear True Wireless™ Accessory (Mini Microphone or TV Streamer) is paired with your processor, use your remote assistant to start and stop streaming audio.

Display shows wireless accessory:



Turn wireless accessory on:



Long-press once.

Turn wireless accessory off:



Short-press once.

Telecoil

Listen to room loops with telecoil.

Turn telecoil on:

Turn telecoil off:



Short-press once.



Short-press once.

Care

Clean your remote control with a soft dry cloth.
Keep your remote control dry.



Hear now. And always



Cochlear Ltd (ABN 96 002 618 073)

1 University Avenue, Macquarie University, NSW 2109, Australia

Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Cochlear Ltd (ABN 96 002 618 073)

14 Mars Road, Lane Cove, NSW 2066, Australia

Tel: +61 2 9428 6555 Fax: +61 2 9428 6352

Cochlear Americas

13059 E Peakview Avenue, Centennial, CO 80111, USA

Tel: +1 303 790 9010 Fax: +1 303 792 9025



Cochlear Deutschland GmbH & Co. KG

Karl-Wiechert-Allee 76A, 30625 Hannover, Germany

Tel: +49 511 542 770 Fax: +49 511 542 7770

株式会社日本コクレア (Nihon Cochlear Co Ltd)

〒113-0033 東京都文京区本郷2-3-7 お茶の水元町ビル

Tel: +81 3 3817 0241 Fax: +81 3 3817 0245

www.cochlear.com

Cochlear, Hear now. And always, Nucleus, and the elliptical logo are either trademarks or registered trademarks of Cochlear Limited.

© Cochlear Limited 2017

D705741 ISS2 SEP17