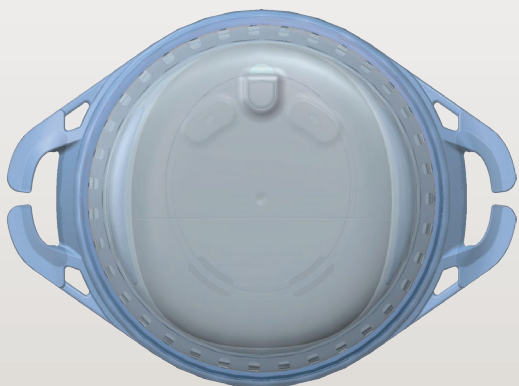
# Aqua+ for Kanso™

User Guide

EN RU UK KK PL HE

CP950

*Hear now. And always*

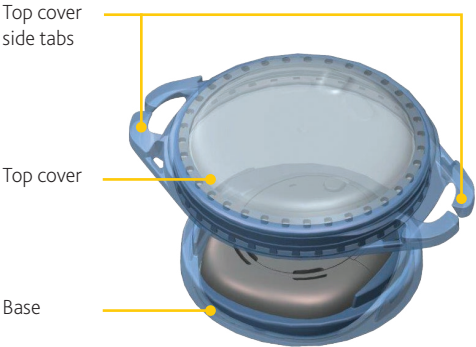
## Contents

<b>EN</b>	Cochlear™ Nucleus® Aqua+ for Kanso — Instructions	1
<b>RU</b>	Инструкции по использованию чехла Cochlear™ Nucleus® Aqua+ для Kanso	17
<b>UK</b>	Cochlear™ Nucleus® Aqua+ для Kanso — інструкція	33
<b>KK</b>	Cochlear™ Nucleus® Aqua+ for Kanso — Нұсқаулар	49
<b>PL</b>	Akcesorium Cochlear™ Nucleus® Aqua+ dla Kanso — instrukcje	65
<b>HE</b>	Cochlear™ Nucleus® Aqua+ הוראות – Kanso של	81

# Aqua+ for Kanso™ Sound Processor

EN

The **Cochlear™ Aqua+ for Kanso™** is a re-usable sealable cover that keeps your Cochlear™ Kanso Sound Processor (model number: CP950) dry during use in or around water (e.g. when swimming, surfing, or just having fun splashing around). It is worn on your implant with your sound processor sealed inside.

The Aqua+ is intended for use by recipients using the Kanso Sound Processor with specific disposable batteries. It should only be used by recipients who can either remove the sound processor themselves if it's causing discomfort or who can indicate any discomfort to their parent or caregiver.

The **Cochlear Headband** is a retention option that holds Aqua+ more securely on your head.

The **Nucleus® Safety Line** is a retention option to help prevent accidental loss of your Aqua+ by securing it to your clothing with a snap-lock clip.

Remotes and wireless accessories can be used with Aqua+, however they should be protected from water and kept dry.

Testing shows Aqua+ can be used at depths of up to 3 metres for up to 2 hours.

Testing shows Aqua+ can be re-used up to 50 times, but this may vary depending on your particular use of it.

There have been no reported undesirable side effects from using Aqua+.

## Battery use with Aqua+

For everyday use, most Kanso Sound Processor users will achieve at least 16 hours battery life with zinc air batteries.

But you cannot use zinc air batteries with Aqua+ as they need air and there is insufficient air inside Aqua+ when it's sealed.

**You must use other disposable batteries for your Kanso Sound Processor to work inside Aqua+. You can use alkaline, nickel metal hydride or silver oxide\* batteries.**

BATTERY TYPE	USE WITH AQUA+	ESTIMATED MINIMUM LIFE
(Zinc air - p675 - PR44)	(No)	(16 hours)
Alkaline (LR44)	Yes	1 hour
Nickel metal hydride (P675)	Yes	1 hour 30 min
Silver oxide (SR44)	Yes	3 hours

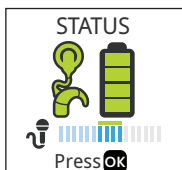
\* (USA) Silver oxide batteries are contraindicated by the FDA.

### NOTE

Battery life depends on your implant type, the thickness of skin over your implant, system settings and hearing situations. It can also vary between battery manufacturers.


If you use a Cochlear Nucleus CR230 Remote Assistant, you can check your remaining battery life (when you're out of the water):

# Aqua+ for Kanso instructions

Before you get wet...

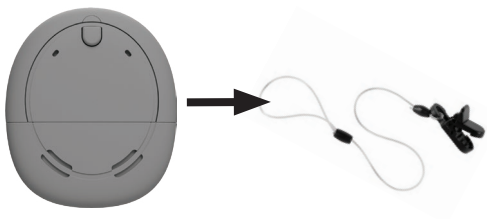1. If you use a Safety Line, remove it from your sound processor (see your sound processor's user guide for instructions).

2. Change your zinc air batteries to a battery type that can be used with Aqua+ (see your sound processor's user guide for instructions on changing batteries).

See page 3 for batteries you can use with Aqua+.

3. Open the Aqua+ housing.  
Put your fingers under the top cover's side tabs and press your thumbs on the raised sides of the base.


**CAUTION**

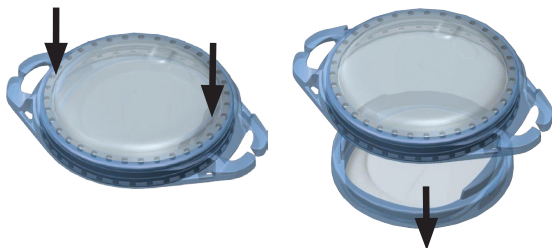Don't damage the soft top cover with your fingernails.

**TIP**

You can gently bend the Aqua+ side tabs towards you, and press alternately on the left then right side until the base pops out.

4. Check both halves for damage (e.g. scratches, tears) or contamination (e.g. sand, hair, sunscreen).  
If it is damaged, use a new Aqua+.

**CAUTION**

Anything that affects the seal may cause water to leak in and damage your sound processor.

5. Place your sound processor **button side up** in the Aqua+ base unit.

**TIPS**

- Your Aqua+ is shaped so your processor only fits in one way.
- If you use a Cochlear SoftWear™ pad, you can leave it on the processor.

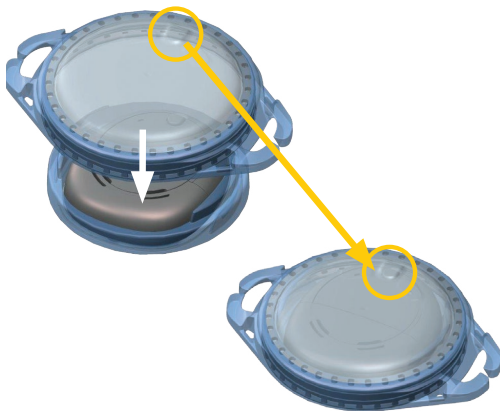
6. Put the Aqua+ top cover on the base unit (**line up the button shape on the top cover** with the button on your sound processor).

**TIP**

Make sure you **don't trap too much air** in the top cover. This may affect sound quality and comfort.

7. Turn the Aqua+ over and **firmly** press around the edges until the base is flush with the top cover.

**CAUTION**

Make sure your Aqua+ is properly sealed, so water doesn't leak in and damage your sound processor.

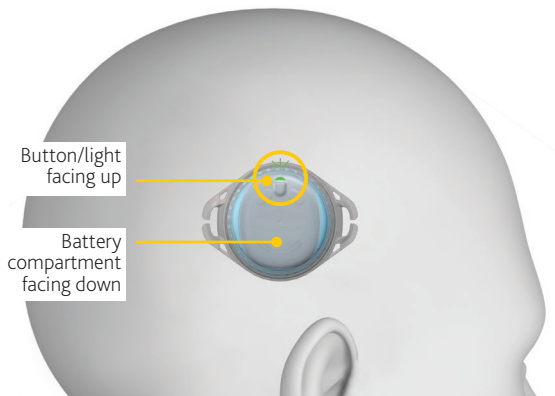
8. If you're worried about losing Aqua+ (e.g. in the surf), use the Safety Line to attach it to your clothing.

See *Safety Line instructions* on page 10.

9. Place the Aqua+ on your implant as normal, making sure the button/light are facing up.

10. Use a retention aid (such as **goggles** as shown here, or the Headband) to keep Aqua+ on your implant.

**IMPORTANT**

Make sure your sound processor is still in the correct position on your implant.

11. Finally, do the **Aqua+ for Kanso Checklist** to make sure you're ready to get wet!

- Using correct batteries (not zinc air)
- Aqua+ tightly sealed
- Safety Line attached if required
- Retention aid ready to use

12. Have fun!

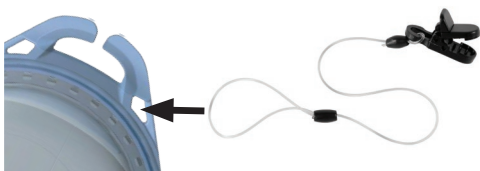
**IMPORTANT**

Don't forget you can't go deeper than 3 metres!

# Safety Line instructions

**⚠ WARNING** Retention lines longer than the Safety Line (standard length) are not recommended for use by children as they may present a risk of strangulation.

1. Thread the loop end of the line through the hole in the Aqua+.

2. Create a knot by passing the clip end of the line through the line loop. Pull it tight.

3. Lift the lever to open the clip. Place the clip over your clothing and press the lever down to close the clip.


After you get wet...

## Care and storage

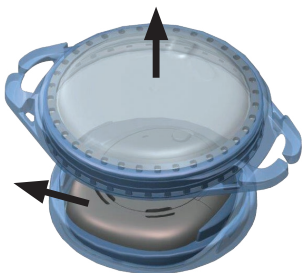1. Rinse Aqua+ with clean water and dry with a soft cloth.
2. Open the Aqua+ top cover and remove your sound processor.

3. Place your Aqua+ in your dry aid kit overnight.
4. Once it's thoroughly dry, put the two halves together and test the seal by pressing down on the top:
  - If any air leaks out, you should not use it again.
5. If the seal is ok, **separate the two halves again** and place your Aqua+ in your storage case or the KANSO Activity Kit's waterproof case until you want to use it again.

## Cautions

- Where possible, avoid pressing the processor button with your fingernail as this can damage Aqua+.
- If your Aqua+ appears damaged or worn, replace it with a new one.
- If your sound processor does get wet, dry it with a soft dry cloth and place it in your dry aid kit for at least eight hours.
- If you use a Softwear pad or Headband, and you experience intermittent sound from your processor, try removing the Softwear pad or Headband.

## Warnings

- If your sound processor becomes unusually warm, remove it immediately and seek advice from your clinician. Parents/caregivers: if your recipient indicates discomfort, check if their sound processor is unusually warm.
- Small parts (e.g. batteries, Safety Line) may be a choking or strangulation hazard.
- If you experience tightness or pain at the implant site, or develop significant skin irritation, stop using your sound processor and contact your clinician.
- If you experience tightness or pain from your retention aid (e.g. Headband, goggles), either loosen it or replace it with a larger retention aid.
- If the coil magnet is too strong, pressure sores may develop at the implant site. If this happens, or if you experience any discomfort in this area, contact your clinician.
- Store spare magnets safely and away from cards that may have a magnetic strip (e.g. credit cards, bus tickets, etc).
- No modification of this equipment is allowed.
- Use only as intended.
- Report unexpected problems to your clinician.

## Other Information

### Environmental conditions

CONDITION	MINIMUM	MAXIMUM
Storage & transport temperature	-10°C (14°F)	+55°C (131°F)
Storage & transport humidity	0% RH	90% RH
Operating temperature	+5°C (41°F)	+40°C (104°F)
Operating relative humidity	0% RH	90% RH
Operating pressure	700 hPa	1060 hPa

### Certification and applied standards

Aqua+ for Kanso Sound Processor fulfils the essential requirements listed in Annex 1 of the EC directive 90/385/EEC on Active Implantable Medical Devices as per the conformity assessment procedure in Annex 2.

The year in which authorisation to affix the CE mark was granted was 2016.

## Warranty

The use of Aqua+ will not impact your Nucleus Global Limited Warranty. Your Nucleus Global Limited Warranty will not be void in circumstances where Aqua+ is used in water with Kanso Sound Processors with non-zinc air batteries only, in accordance with the Aqua+ instructions.

## Legal statement

The statements made in this guide are believed to be true and correct as of the date of publication. However, specifications are subject to change without notice.

© Cochlear Limited 2016

# Labelling symbols

The following symbols may appear on Aqua+ components and/ or packaging.

Refer to instruction manual

Specific warnings or precautions associated with the device, which are not otherwise found on the label

Manufacturer

Authorised representative in the European Community

Catalogue number

Serial number

Batch code

Date of manufacture

Temperature limits

CE registration mark with notified body number

**Rx Only**

By prescription

Recyclable material

Dispose of electrical components in accordance with your local regulations

Ingress Protection Rating:

**IP68**

- Protected against failure from dust penetration
- Protected against failure from continuous immersion in water up to 3 metres deep for up to 2 hours

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**Cochlear Canada Inc** 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada  
Tel: +1 416 972 5082 Fax: +1 416 972 5083

**Cochlear AG** EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland  
Tel: +41 61 205 8204 Fax: +41 61 205 8205

**ECREP Cochlear Deutschland GmbH & Co. KG** Karl-Wiechert-Allee 76A, 30625 Hannover, Germany  
Tel: +49 511 542 770 Fax: +49 511 542 7770

**Cochlear Europe Ltd** 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey KT15 2HJ, United Kingdom  
Tel: +44 1932 26 3400 Fax: +44 1932 26 3426

**Cochlear Benelux NV** Schaliënhoedreef 20 i, B-2800 Mechelen, Belgium  
Tel: +32 15 79 55 11 Fax: +32 15 79 55 70

**Cochlear France S.A.S.** 135 Route de Saint-Simon, 31035 Toulouse, France  
Tel: +33 5 34 63 85 85 (International) or 0805 200 016 (National) Fax: +33 5 34 63 85 80

**Cochlear Italia S.r.l.** Via Larga 33, 40138 Bologna, Italy  
Tel: +39 051 601 53 11 Fax: +39 051 39 20 62

**Cochlear Nordic AB** Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden  
Tel: +46 31 335 14 61 Fax: +46 31 335 14 60

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**Cochlear (HK) Limited** Room 1204, 12/F, CRE Building, No 303 Hennessy Road, Wanchai, Hong Kong SAR  
Tel: +852 2530 5773 Fax: +852 2530 5183

**Cochlear Korea Ltd** 1st floor, Cheongwon Building 33, Teheran-ro 8 gil, Gangnam-gu, Seoul, Korea  
Tel: +82 2 533 4450 Fax: +82 2 533 8408

**Cochlear Medical Device (Beijing) Co., Ltd**

Unit 2208 Gemdale Tower B, 91 Jianguo Road, Chaoyang District, Beijing 100022, P.R. China  
Tel: +86 10 5909 7800 Fax: +86 10 5909 7900

**Cochlear Medical Device Company India Pvt. Ltd.**

Ground Floor, Platina Building, Plot No C-59, G-Block, Bandra Kurla Complex, Bandra (E), Mumbai – 400 051, India  
Tel: +91 22 6112 1111 Fax: +91 22 6112 1100

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**Cochlear Middle East FZ-LLC**

Dubai Healthcare City, Al Razi Building 64, Block A, Ground Floor, Offices IR1 and IR2, Dubai, United Arab Emirates  
Tel: +971 4 818 4400 Fax: +971 4 361 8925

**Cochlear Latinoamérica S.A.**

International Business Park, Building 3835, Office 403, Panama Pacifico, Panama  
Tel: +507 830 6900 Fax: +507 830 6218

**Cochlear NZ Limited**

Level 4, Takapuna Towers, 19-21 Como St, Takapuna, Auckland 0622, New Zealand  
Tel: +64 9 914 1983 Fax: 0800 886 036

[www.cochlear.com](http://www.cochlear.com)

ACE, Advance Off-Stylet, AOS, AutoNRT, Autosensitivity, Beam, Button, Carina, Cochlear, コクレア, Codacs, Contour, Contour Advance, Custom Sound, ESPrit, Freedom, Hear now. And always, Hybrid, inHear, Invisible Hearing, MET, MP3000, myCochlear, NRT, Nucleus, 科利耳, Off-Stylet, SmartSound, Softip, SoftWear, SPrint, the elliptical logo and Whisper are either trademarks or registered trademarks of Cochlear Limited. Ardium, Baha, Baha Divino, Baha Intenso, Baha PureSound, Baha SoftWear, BCDrive, DermaLock, Vistafix and WindShield are either trademarks or registered trademarks of Cochlear Bone Anchored Solutions AB. Bluetooth is a registered trademark of Bluetooth SIG.

© Cochlear Limited 2016

D801181 ISS1 JUL16

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**Cochlear Tibbi Cihazlar ve Sağlık Hizmetleri Ltd. Şti.**