



## Benefits of your Cochlear™ Wireless TV Streamer

The Cochlear Wireless TV Streamer will provide you with high quality streamed audio from your TV, stereo, personal computer or similar device directly to your sound processors.

*Note: The TV streamer can be paired with several sound processors to facilitate multiple users listening to the same streamers or for bilateral or bimodal use.*

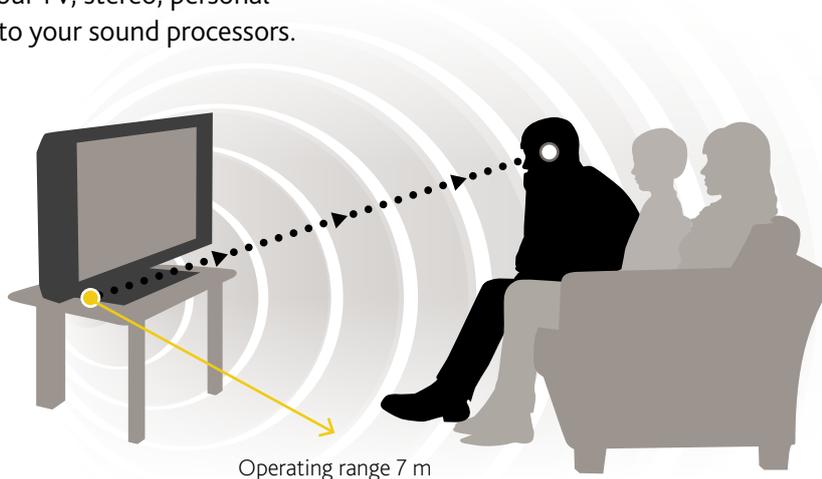

## Product overview

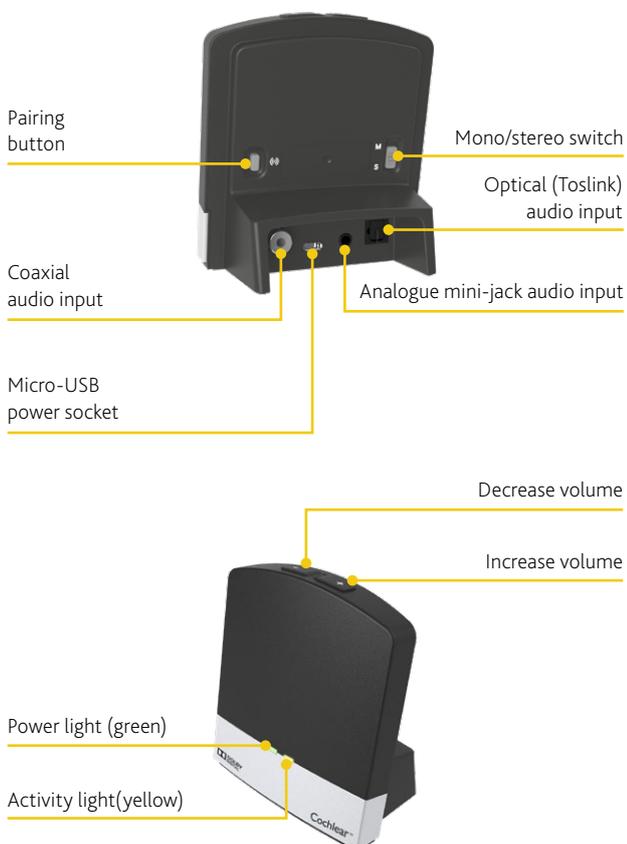

## Get started

Check with your hearing healthcare professional whether your sound processor has been paired with your TV Streamer. Previously paired sound processors do not need to be paired again. If your sound processor is not paired, see below for instructions on how to easily pair it with your TV Streamer.

The TV Streamer supports both analogue and digital audio inputs. Depending on your TV, HiFi stereo, computer or other audio device, the TV Streamer can be installed in several ways, as described on the following pages.


For all installations, it is necessary to:

- A** Plug the power cord into the micro-USB socket on the TV Streamer.
- B** Plug the other end of the power cord into an electric wall socket. Connect the TV Streamer to your audio source using one of the following 5 installations.

## Set up of the Cochlear Wireless TV Streamer

### Connecting the TV Streamer to an analogue audio source (e.g. TV or HiFi stereo).


- A** Plug the stereo mini-jack on the audio cable into the stereo mini-jack input on the TV Streamer.
- B** Plug the other end of the audio cord into the "AUDIO OUT" terminals on the audio source. "AUDIO OUT" is usually marked "L – R" for left and right, and is normally located on the back of the audio source.

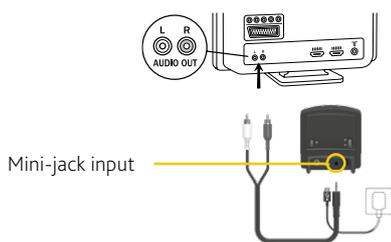

Mini-jack input

### Connecting the TV Streamer to an analogue audio source using a headphone mini-jack adaptor.


- A** Plug the stereo mini-jack on the audio cable into the stereo mini-jack input on the TV Streamer.
- B** Connect the stereo mini-jack adaptor to the red and white plugs at the other end of the audio cable.
- C** Plug the stereo mini-jack adaptor into the audio source's "HEADPHONE" output terminal.

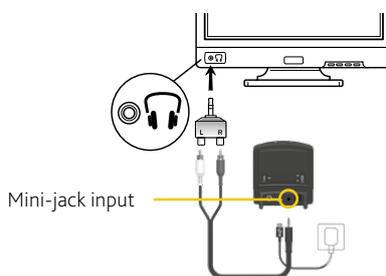

Mini-jack input

### Connecting the TV Streamer to an analogue audio source using a scart adaptor.


- A** Plug the stereo mini-jack on the audio cable into the stereo mini-jack input on the TV Streamer.
- B** Connect the scart adaptor to the red and white plugs on the other end of the audio cable.
- C** Plug the scart adaptor into the audio source's scart terminal.

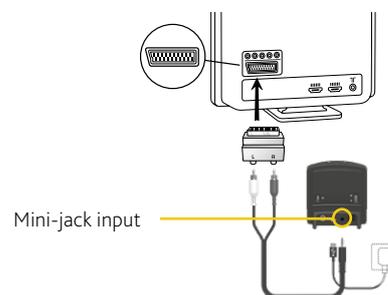

Mini-jack input

### Connecting the TV Streamer to a digital audio source using the optical (Toslink) audio input.



- A** Insert one of the plugs on the optical audio cable into the optical audio input socket on the TV Streamer.
- Note:** Most optical cables are delivered with a small protective cap to protect the tip of the plug. Please remove this protective cap before inserting the cable.
- B** Plug the other end of the optical audio cable into the "Digital Audio Out/Optical" socket on your audio source.

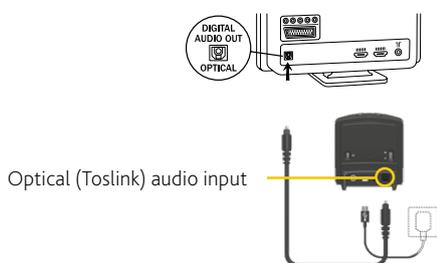

Optical (Toslink) audio input

### Connecting the TV Streamer to a digital audio source using the coaxial audio input.

- A** Insert one of the plugs on the coaxial audio cable into the coaxial audio input socket on the TV Streamer.
- B** Plug the other end of the coaxial audio cable into the "Digital Audio Out/ Coaxial" socket on your audio source.

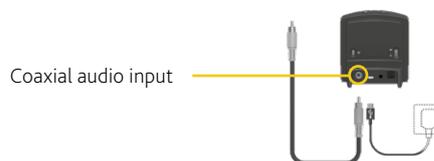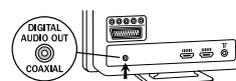

Coaxial audio input


# Quick guide for Baha® and Osia® sound processor users

## Cochlear™ Wireless TV Streamer



### Pair with sound processor


- A** Make sure the TV Streamer is plugged into a wall socket with the green power light on.

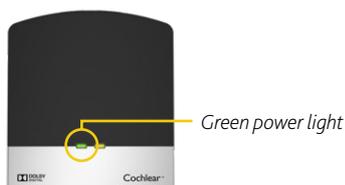

- B** Turn off your sound processor.
- C** Press the pairing button on the TV streamer once to activate its pairing mode.



- D** You can tell the pairing mode has been activated when the channel indicator is flashing. Pairing mode will be active for 20 seconds.

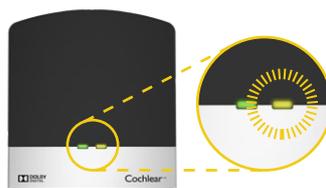

- E** While the channel indicator light is flashing, turn on your sound processor.

**Note:** Successful pairing will be indicated by an audible melody played in your sound processor(s).

You are now ready to use the TV Streamer!

### Pair several TV Streamers to one sound processor

It is possible to pair up to three TV Streamer units with your sound processor(s).

To pair a second or third TV Streamer, follow steps **A** and **B** left of this column.

- C** Then press the pairing button of the second TV Streamer **two** or **three** times.
- D** The yellow light indicator on the front of the TV Streamer will now blink two or three times every 2 seconds to indicate it is ready to be paired in Channel 2 or 3 of the sound processor.

Pairing button pushes	LED blinking pattern	Channel
1	1 1 1 1	1
2	1 2 1 2 1 2 1 2	2
3	1 2 3 1 2 3 1 2 3 1 2 3	3

- E** Complete the pairing process by turning on the sound processor(s).

### Using the Cochlear Wireless TV Streamer

#### Start streaming

After installation, you can listen to streamed audio from the TV Streamer by following these simple steps:

First, switch on your TV, sound system or computer and then, press and hold down the button (long

press) on your sound processor(s). If necessary, this must be done separately for each processor in order to hear streamed audio in both ears.

#### Stop streaming

Briefly press and release the button on your sound processor.

Cochlear Wireless Accessories are compatible with select Cochlear sound processors. See [www.cochlear.com/compatibility](http://www.cochlear.com/compatibility) for full compatibility details. Not all products are available in all markets. Product availability is subject to regulatory approval in respective markets.

Quick guide for Baha® and Osia® sound processor users

# Cochlear™ Wireless TV Streamer



## Support

For full details about the Cochlear Wireless TV Streamer, please refer to the user manual.

For full details about the Cochlear sound processors, please refer to their user manual.



Visit [www.cochlear.com](http://www.cochlear.com) to learn about our full range of apps and wireless accessories.

 **Cochlear Bone Anchored Solutions AB**, Konstruktionsvägen 14,  
435 33 Mölnlycke, Sweden  
Tel: +46 31 792 44 00, Fax: +46 31 792 46 95

[www.cochlear.com](http://www.cochlear.com)

Please seek advice from your health professional about treatments for hearing loss. Outcomes may vary, and your health professional will advise you about the factors which could affect your outcome. Always read the instructions for use. Not all products are available in all countries. Please contact your local Cochlear representative for product information.

Cochlear, 科利耳, コクレア, 코클리어, Hear now. And always, Baha, Osia, the elliptical logo, and marks bearing an ® or ™ symbol, are either trademarks or registered trademarks of Cochlear Limited or Cochlear Bone Anchored Solutions AB (unless otherwise noted).  
Dolby and the double-D symbol are registered trademarks of Dolby Laboratories.

© Cochlear Bone Anchored Solutions AB 2019. All rights reserved. OCT19.

D1661031-V1