

# Cochlear™ Baha® Remote Control 2


## Benefits of your Cochlear Baha Remote Control 2



- Allows you to discreetly adjust your Baha 4 Sound Processor
- Provides large buttons for simple volume or program changes
- Allows you to monitor your or your child's sound processor
- Makes streaming to other wireless accessories easy


**Note:** The Baha Remote Control 2 can be paired to two Baha 4 Sound Processors for those with a bilateral fitting.

## Product overview

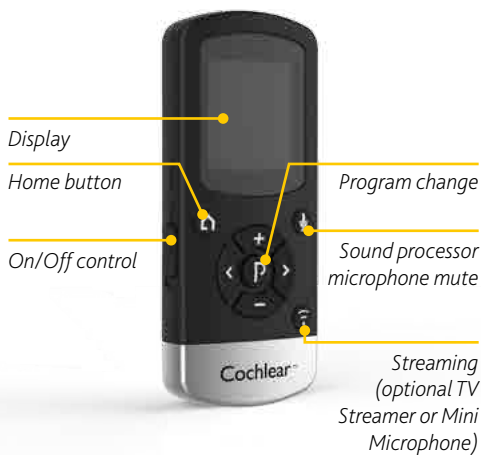

## Getting started

Pre-paired sound processors do not need to be paired again. If your sound processor is not pre-paired, instructions on how to easily pair it with your Remote Control 2 are on the last page of this guide.

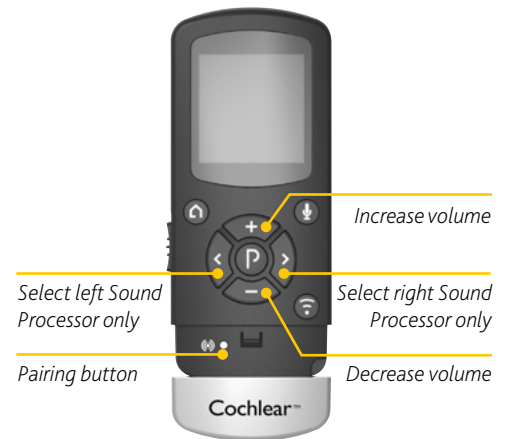

## Charging the battery

Before using your Remote Control 2, you need to fully charge the battery. To charge it, plug in the charging cable as shown to the right. Then plug the other end into a power outlet. During charging the bars in the battery meter will be showing one, two and three bars continuously. When fully charged all three bars will be shown continuously.



# Cochlear™ Baha® Remote Control 2

## Using the Remote Control 2


### Turning your Remote Control 2 ON and OFF

- To turn your Remote Control 2 ON slide the switch on the side of the Remote Control 2 down to the ON position. A white "I" icon will be visible on the base of the switch and the display will turn on.
- To turn your Remote Control 2 OFF, slide the ON/OFF switch up to the OFF position.


#### Activate streaming



If your sound processor is paired with more than one wireless accessory press once, twice, or three times to toggle between the accessories. If you cannot connect to your wireless accessory, move closer to it and try again.

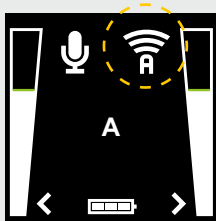

Connected to streaming device.

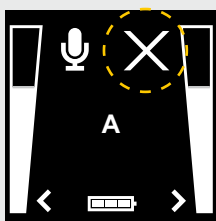


No streaming device found.


#### Switching between sides



Select your left or right sound processor if you use 2 sound processors.

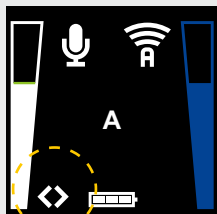

Left and right hearing instrument volume selected.

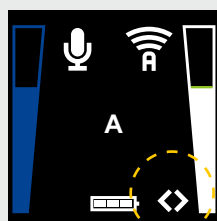

Left and right streaming volume selected.

#### Deactivating streaming



Home: Stop streaming and return to program 1 and default volume settings


Stop streaming and return to the last used program and volume settings

#### Remote Control 2 displays

The batteries are low in the Baha Sound Processor and need to be replaced:




In the left sound processor.

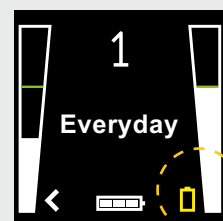

In the right sound processor.



The battery is low in the Remote Control 2 and need to be charged.

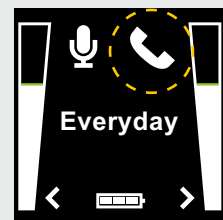

The Phone Clip is active.

## Support

For full details about the Cochlear Baha Remote Control 2, please refer to the User Manual.

For full details about the Cochlear Baha 4 Sound Processor, please refer to the User Manual part A.

For further support and videos, visit the Cochlear website: [www.cochlear.com](http://www.cochlear.com)

# Cochlear™ Baha® Remote Control 2

## Pairing with your Baha Sound Processor

Make sure the Remote Control 2 is fully charged and is turned ON.

**A** Switch off your Baha 4 Sound Processor:




**B** Press the pairing button on the Remote Control 2 once using the tip of a pen or similar object.



**C** The 'search' icon will display on the screen and the Remote Control will now be in pairing mode for 20 seconds.

Searching for both hearing instruments.

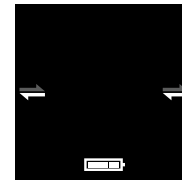

**D** While pairing mode is active, you can switch on your sound processor in two different ways:



**E** Successful pairing will be indicated by an audible melody played in the sound processor.



**F** Searching and connecting to the hearing instruments

Connected to left hearing instrument and searching for right hearing instrument



Not everyone with hearing loss is a candidate for a Baha. All surgical procedures include an element of risk, and it is impossible to guarantee success. For complete information regarding the risks and benefits of a Baha procedure, please refer to the Instructions for use for the Baha implant available at [www.Cochlear.com/US/BahaIndications](http://www.Cochlear.com/US/BahaIndications)

[www.Cochlear.com/US](http://www.Cochlear.com/US)

**Cochlear Americas**  
13059 East Peakview Avenue  
Centennial, CO 80111 USA  
Telephone: 1 303 790 9010  
Support: 1 800 483 3123

**Cochlear Canada Inc.**  
2500-120 Adelaide Street West  
Toronto, ON M5H 1T1 Canada  
Telephone: 1 416 972 5082  
Fax: 1 416 972 5083

BUN300 ISS1 AUG14

©Cochlear Limited 2014. All rights reserved. Hear now. And always and other trademarks and registered trademarks are the property of Cochlear Limited or Cochlear Bone Anchored Solutions AB. The names of actual companies and products mentioned herein may be the trademarks of their respective owners.

Hear now. And always

