95147

**Manufacturer:**

Cochlear Bone Anchored Solutions AB Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden Tel: +46 31 792 44 00 Fax: +46 31 792 46 95

**Regional offices:**

Cochlear Ltd (ABN 96 002 618 073) 1 University Avenue, Macquarie University, NSW 2109, Australia Tel: +61 2 9428 6555 Fax: +61 2 9428 6352  
 Cochlear Americas 13059 E Peakview Avenue, Centennial, CO 80111, USA Tel: +1 303 790 9010 Fax: +1 303 792 9025  
 Cochlear AG EMEA Headquarters, Peter Merian-Weg 4, 4052 Basel, Switzerland Tel: +41 61 205 0404 Fax: +41 61 205 0405

**Local offices:**

Cochlear Deutschland GmbH & Co. KG Karl-Wiechert-Allee 76A, 30625 Hannover, Germany Tel: +49 511 542 770 Fax: +49 511 542 7770  
 Cochlear Europe Ltd 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey, KT15 2HJ, United Kingdom Tel: +44 1932 26 3400 Fax: +44 1932 26 3426  
 Nihon Cochlear Co Ltd Ochanomizu-Motomachi Bldg, 2-3-7 Hongo, Bunkyo-Ku, Tokyo 113-0033, Japan Tel: +81 3 3817 0241 Fax: +81 3 3817 0245  
 Cochlear (HK) Limited Unit 1204, 12/F, 297-307 Hennessy Road, Wan Chai, Hong Kong Tel: +852 2530 5773 Fax: +852 2530 5183  
 Cochlear Medical Device (Beijing) Co Ltd Unit 2208-2212, Gemdale Tower B, 91 Jianguo Road, Chaoyang District, Beijing 100022, PR China  
 Tel: +86 10 5909 7800 Fax: +86 10 5909 7900  
 Cochlear Limited (Singapore Branch) 6 Sin Ming Road, #01-16 Sin Ming Plaza Tower 2, Singapore 575585 Tel: +65 6553 3814 Fax: +65 6451 4105  
 Cochlear Korea Ltd 1st floor, Cheongwon building, 828-5, Yuksam dong, Kangnam gu, Seoul, Korea Tel: +82 2 533 4663 Fax: +82 2 533 8408  
 Cochlear Benelux NV Schaliënhoefdreef 20 I, 2800 Mechelen, Belgium Tel: +32 1579 5511 Fax: +32 1579 5500  
 Cochlear Medical Device Company India PVT Ltd Platina Bldg, Ground Floor, Plot No. C 59, G Block, BKC, Bandra East, Mumbai 400051, India  
 Tel: +91 22 6112 1111 Fax: +91 22 61121100  
 Cochlear Italia SRL Via Larga 33, 40138 Bologna, Italy Tel: +39 051 601 53 11 Fax: +39 051 39 20 62  
 Cochlear France S.A.S. Route de l'Orme aux Merisiers, ZI les Algorithmes – Bât. Homère, 91190 Saint-Aubin, France Tel: +33 805 200 016 Fax: +33 160 196 499  
 Cochlear Nordic AB Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden Tel: +46 31 335 14 61 Fax: +46 31 335 14 60  
 Cochlear Tibbi Cihazlar ve Sağlık Hizmetleri Ltd. Sti. Cubuklu Mah. Bogazici Cad. Bogazici Plaza No: 6/1 Kavacik, 34805 Beykoz-Istanbul, Turkey  
 Tel: +90 216 538 5900 Fax: +90 216 538 5919  
 Cochlear Latinoamerica, S. A. International Business Park Building 3835, Office 103, Panama Pacifico, Panama Tel: +507 830 6220 Fax: +507 830 6218  
 Cochlear Canada Inc 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada Tel: +1 416 972 5082 Fax: +1 416 972 5083

[www.cochlear.com](http://www.cochlear.com)

Ardium, Baha, Baha Divino, Baha Intenso, Baha PureSound, Baha SoftWear, BCDrive, DermaLock, Vistafix and WindShield are either trademarks or registered trademarks of Cochlear Bone Anchored Solutions AB. Cochlear, Hear now. And always and the elliptical logo are either trademarks or registered trademarks of Cochlear Limited. The Bluetooth word mark and logos are registered trademarks owned by Bluetooth SIG, Inc. and any use of such marks by Cochlear Limited is under license. Cochlear Baha 5 Sound Processor is compatible with iPhone 6 Plus, iPhone 6, iPhone 5s, iPhone 5c, iPhone 5, iPad Air 2, iPad Air, iPad (4th generation), iPad mini 3, iPad mini 2, iPad mini and iPod touch (5th generation) using iOS 7.1.2, iOS 8.1 or later. Apple, the Apple logo, AirPlay, FaceTime, Live Listen, iPhone, iPad and iPod are trademarks of Apple Inc., registered in the U.S. and other countries. Android is a trademark of Google Inc. © Cochlear Bone Anchored Solutions AB 2015. All rights reserved. JAN15. Printed in Sweden

"Made for iPhone" means that an electronic accessory has been designed to connect specifically to iPhone and has been certified by the developer to meet Apple performance standards. Apple is not responsible for the operation of this device or its compliance with safety and regulatory standards. Please note that the use of this accessory with iPhone may affect wireless performance.

0413

622852-00

# Cochlear™ Baha® 5 Sound Processor

## User manual part A

 English Français Español Português **ZONE 8**

### Sound processor type designations for models included in this User Manual are:

FCC ID: QZ3BAHA5, IC: 8039C-BAHA5, IC model: Baha® 5.

**Statement:**

This device complies with Part 15 of the FCC Rules.

Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

**Note:** This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from the one in which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

• Changes or modifications can void the user's authority to operate the equipment.

**Intended use**

The Cochlear™ Baha® 5 Sound Processor uses bone conduction to transmit sounds to the cochlea (inner ear). It works by combining a sound processor and a small titanium implant that is placed in the skull behind the ear. The skull bone integrates with the titanium implant through a process called osseointegration. This allows sound to be conducted via the skull bone directly to the cochlea, which improves hearing performance.

**List of countries:**

Not all products are available in all markets. Product availability is subject to regulatory approval in the respective markets.

The products are in compliance with the following regulatory requirements:

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Equipment includes RF transmitter.

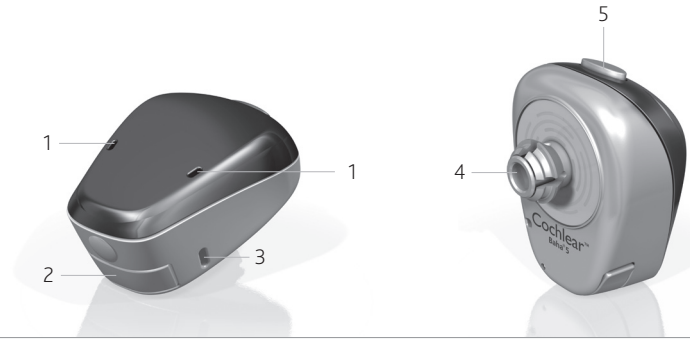
Hear now. And always


Cochlear®

1

2

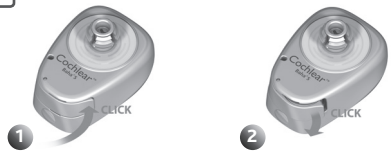3

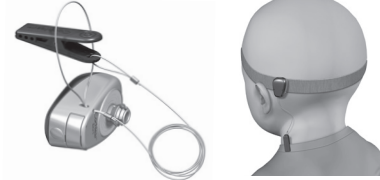
4

5

6

7

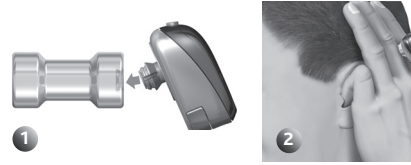
8


9

10

11

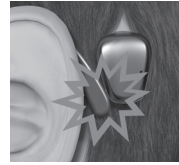Horizontal lines for notes:

# Welcome

Congratulations on your choice of the Cochlear™ Baha® 5 Sound Processor. You are now ready to use Cochlear's highly advanced bone conduction sound processor, featuring sophisticated signal processing and wireless technology.

This manual is full of tips and advice on how to best use and care for your Baha sound processor. By reading this manual and then keeping it handy for future reference, you will ensure that you get the most benefit out of your Baha sound processor.

## Key to device *See figure* **1**

1. Microphones
2. Battery compartment door
3. Attachment point for the safety line
4. Plastic snap connector
5. Program button, Wireless audio streaming button

**Note on figures:** The figures included on the cover correspond to information specific to this model of sound processor. Please reference the appropriate figure when reading. The images shown are not to scale.

## Introduction

To ensure optimal performance, your hearing care professional will fit the processor to suit your needs. Be sure to discuss any questions or concerns that you may have regarding your hearing or use of this system with your hearing care professional.

## Warranty

**The warranty does not cover defects or damage arising from, associated with, or related to the use of this product with any non-Cochlear processing unit and/or any non-Cochlear implant. See the "Cochlear Baha Global Limited Warranty card" for more details.**

## Contacting Customer Service

We strive to provide you with the best possible products and services. Your views and experiences with our products and services are important to us. If you have any comments you would like to share, please contact us.

### Customer Service – Cochlear Americas

13059 E. Peakview Avenue,  
Centennial, CO 80111, USA

Toll free (North America) 1800 523 5798

Tel: +1 303 790 9010, Fax: +1 303 792 9025  
E-mail: [customer@cochlear.com](mailto:customer@cochlear.com)

### Customer Service – Cochlear Europe

6 Dashwood Lang Road  
Bourne Business Park, Addlestone,  
Surrey KT15 2HJ, United Kingdom

Tel: +44 1932 26 3400, Fax: +44 1932 26 3426  
E-mail: [info@cochlear.co.uk](mailto:info@cochlear.co.uk)

### Customer Service – Cochlear Asia Pacific

1 University Avenue, Macquarie University  
NSW 2109, Australia

Toll free (Australia) 1800 620 929  
Toll free (New Zealand) 0800 444 819

Tel: +61 2 9428 6555, Fax: +61 2 9428 6352  
or Toll free Fax 1800 005 215  
E-mail: [customerservice@cochlear.com.au](mailto:customerservice@cochlear.com.au)

## Key to symbols

The following symbols will be used throughout this document. Please refer to the list below for explanations:

	"Consult instructions for use" or "Operating instructions"
	"Caution" or "Caution, consult accompanying documents"
	Audible signal
	CE-mark
	Temperature limitation
	Interference risk
	Product name, CE mark and ACMA symbol
	Batch Code
	Serial Number
	Refer to instructions/booklet
	ACMA (Australian Communications and Media Authority) compliant
	Made for iPod®, iPhone®, iPad®
	Bluetooth® Smart

## Using your sound processor

The button on your sound processor lets you choose from your pre-set programs and enable/disable wireless streaming. You can choose to enable the audio indicators to alert you of changes to the settings and processor status.

Your sound processor is programmed to be used as either a left or right-sided device. Your hearing care professional will have marked your processor(s) with a L or R indicator.

If you are a bilateral user, changes you make to one device will automatically apply to the second device.

### On/off *See figure 2*

- 1 Turn on your sound processor by completely closing the battery compartment.
- 2 Turn off your sound processor by gently opening the battery compartment until you feel the first "click".

When your sound processor is turned off and then back on again, it will return to the default setting (program one).

### Audible status indicators *See figure 3*

Your sound processor is equipped with audible indicators. For an overview of the audible indicators, refer to the table at the back of this section.

Your hearing care professional can disable the audio indicators if you prefer.

### Changing the program *See figure 4*

Together with your hearing care professional you will have selected up to four pre-set programs for your sound processor:

**Program 1:** \_\_\_\_\_

**Program 2:** \_\_\_\_\_

**Program 3:** \_\_\_\_\_

**Program 4:** \_\_\_\_\_

These programs are suitable for different listening environments. Ask your hearing care professional to fill in your specific programs.

To switch programs, press and release the button on your sound processor. If enabled, an audio indicator will let you know which program you are using:

**Program 1:** 1 beep

**Program 2:** 2 beeps

**Program 3:** 3 beeps

**Program 4:** 4 beeps

## Volume settings

Your hearing care professional has set the volume level for your sound processor.

**You can adjust the volume level with the optional Cochlear Baha Remote Control, the Cochlear Wireless Phone Clip or an iOS device (see Made for iPhone section).**

### Using the telephone *See figure 5*

For best results when talking on the telephone, place the receiver near the microphone inlet of your sound processor instead of near your ear. Make sure the receiver does not touch the sound processor, as this may cause feedback.

Your sound processor is a Made for iPhone® hearing device (MFi). This lets you connect your sound processor directly with a compatible iPhone. When connected, you can stream audio directly from your iPhone to your sound processor.

To stream wirelessly from other mobile and cordless phones you can use the optional Cochlear Wireless Phone Clip. You will need to check the compatibility of your phone with this product.

For assistance in pairing and using mobile and cordless phones with your sound processor, please contact your hearing care professional or visit: [www.cochlear.com](http://www.cochlear.com).

## Made for iPhone (MFi)

Your sound processor is a Made for iPhone (MFi) hearing device. This allows you to control your sound processor and stream audio directly from an iPhone®, iPad® or iPod touch®. Ask your hearing care professional or visit: [www.cochlear.com](http://www.cochlear.com) for more information.

## Wireless accessories

You can use Cochlear Wireless accessories to enhance your listening experience. Ask your hearing care professional to learn more about your options or visit [www.cochlear.com](http://www.cochlear.com).

**To activate wireless audio streaming**, press and hold the sound processor button until you hear a melody. See figure 4

**To end wireless audio streaming**, press and release the button. The sound processor will return to the previous program.

## Flight mode See figure 8

When boarding a flight, wireless functionality must be deactivated because radio signals may not be transmitted during flights. To disable wireless operation:

1. Turn off the sound processor by opening the battery compartment.
2. Press the button and close the battery compartment at the same time.

To deactivate flight mode, turn the sound processor off and back on again. (by opening and closing the battery compartment).

## Changing batteries

When there is approximately one hour of battery power remaining, you will hear an audio indicator comprising of two sets of four short beeps. At this time you may experience low amplification. If the battery runs down completely, the sound processor will beep and then stop working.

Use one of the included batteries (zinc-air, non-rechargeable) as a replacement. The batteries included in the sound processor kit reflect Cochlear's latest recommendations. Contact your hearing care professional for additional batteries.

### To replace the battery:

See figure 7

- 1 Hold the sound processor with the back facing upwards.
- 2 Gently open the battery compartment until it is completely open. Remove the old battery. Dispose of the battery according to local regulations. Remove the sticker on the + side of the new battery. Insert the new battery with the + sign facing upwards in the battery compartment.
- 3 Gently close the battery compartment until it is completely closed.

### Battery tips

- Battery life decreases as soon as the battery is exposed to air (when the plastic strip has been removed).
- Battery life depends on daily use, the volume setting, the use of wireless audio streaming, the sound environment, the program setting, and battery strength.
- To maximise battery life, switch off the sound processor when its not in use.
- If a battery leaks, replace it immediately.

## Attaching the safety line *See figure 6*

Before connecting your sound processor, attach the safety line and clip it to your shirt or jacket.

Cochlear recommends connecting the safety line when engaging in physical activities. Children should use the safety line at all times.

## Optional tamper-proof battery door *See figure 9*

To prevent the accidental opening of the battery door, an optional tamper-proof battery door is available. This is particularly useful for children. Contact your hearing care professional for tamper-proof battery doors.

### To use the tamper-proof battery door:

*See figure 9*

1. To unlock the device, carefully insert the tip of a pen in the small hole on the battery door and gently open the battery compartment.
2. To lock the device, gently close the battery compartment until it is completely closed.

 **Caution:** Batteries can be harmful if swallowed. Be sure to keep your batteries out of reach of small children and household pets. In the event a battery is accidentally swallowed, seek immediate medical attention at the nearest emergency centre.

## General processor care

Your Baha sound processor is a delicate electronic device. Follow these guidelines to keep it in proper working order:

- When not in use, turn off your sound processor and store it free from dust and dirt.
- If you will not be using your sound processor for a long period, remove the battery.
- During physical activities, secure your sound processor by using the safety line.
- Remove your sound processor before applying hair conditioners, mosquito repellent and or similar products.

## Using Share the Experience

*See figure 10*

Family members and friends can “share the experience” of bone conduction hearing. The test rod can be used by others to hear with the sound processor.

### To use the test rod:

- 1 Turn on your sound processor and snap it onto the test rod using the tilt technique. Hold the rod against the skull bone behind an ear. Plug both ears and listen.
- 2 To avoid feedback (whistling), the sound processor should not touch anything other than the test rod.

## Warnings

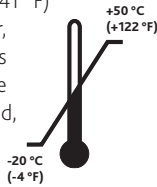
### General advice

A sound processor will not restore normal hearing and will not prevent or improve a hearing impairment resulting from organic conditions.

- Infrequent use of a sound processor may not enable a user to attain full benefit from it.
- The use of a sound processor is only part of hearing rehabilitation and may need to be supplemented by auditory and lip reading training.

### Warnings

- The sound processor is a digital, electrical, medical instrument designed for specific use. As such, due care and attention must be exercised by the user at all times.
- The sound processor is not waterproof!
- Never wear it in heavy rain, in the bath or shower!
- Do not expose the sound processor to extreme temperatures. It is designed to operate within the temperature ranges +5 °C (+41 °F) to +40 °C (+104 °F). In particular, battery performance deteriorates in temperatures under +5 °C. The processor should not be subjected, at any time, to temperatures below -20 °C (-4 °F) or above +50 °C (+122 °F).
- Contains small parts that may present a choking hazard.
- This product is not suitable for use in flammable and/or explosive environments.
- If you are to undergo an MRI (Magnetic Resonance Imaging) procedure, refer to the MRI Reference Card included in the document pack.
- Portable and mobile RF (radio frequency) communications equipment can affect the performance of your sound processor.

- The sound processor is suitable for use in electromagnetic environments with mains power of typical commercial or hospital quality, and power frequency magnetic fields of typical commercial or hospital levels. It is intended for use in an electromagnetic environment in which radiated RF (radio frequency) disturbances are controlled.

- Interference may occur in the vicinity of equipment with the symbol to the right.
- Dispose of batteries and electronic items in accordance with your local regulations.
- Discard your device as electronic waste according to local regulations.
- When the wireless function is activated, the sound processor uses low-powered digitally coded transmissions in order to communicate with other wireless devices. Although unlikely, nearby electronic devices may be affected. In that case, move the sound processor away from the affected electronic device.
- When using wireless functionality and the sound processor is affected by electromagnetic interference, move away from the source of this interference.
- Be sure to deactivate wireless functionality when boarding flights.
- Turn off your wireless functionality by using the flight mode in areas where radio frequency emission is prohibited.
- Cochlear Baha wireless devices include an RF transmitter that operates in the range of 2.4 GHz–2.48 GHz.
- For wireless functionality, only use Cochlear Wireless accessories. For further guidance regarding e.g. pairing, please refer to the user guide of the relevant Cochlear Wireless accessory.
- No modification of this equipment is allowed.

**If the sound processor becomes very wet:**

1. Immediately open the battery door and remove the battery.
2. Put your sound processor in a container with drying capsules such as a Dri-Aid Kit, etc. Leave it to dry out overnight. Drying kits are available from most hearing care professionals.

**Feedback (whistling) problems**

See figure 11

- 1 Check to ensure that your sound processor is not in contact with items such as glasses or a hat, since that can cause feedback. Also ensure the sound processor is not in contact with your head or ear.
- 2 Check that the battery compartment is closed.
- 3 Check that there is no external damage to the sound processor.

Functions and audible indicators		
Function	On the device	Audible indicator
Turn ON	Close battery compartment	10 beeps
Turn OFF	Open battery compartment	None
Change program	Short button press	1 beep for program 1 2 beeps for program 2 etc
Low battery warning		4 beeps 2 times
When using Cochlear Wireless Accessory or MFi		
Change volume	With Cochlear Baha Remote Control, Cochlear Wireless Phone Clip or with an iPhone	1 beep indicates volume increase/ decrease
Volume limit reached	With Cochlear Baha Remote Control, Cochlear Wireless Phone Clip or with an iPhone	1 long beep
Enable wireless audio streaming	Long button press	5 beeps (melody in upward scale)
Disable wireless audio streaming	Short button press	1 beep for program 1 2 beeps for program 2 etc
Pairing confirmation	None	5 beeps (melody in upward scale)
Toggle between wireless accessories	Long button press	None
Turn ON flight mode	Press button and close battery compartment	10 beeps 2 times

*Note: Your hearing care professional may have disabled some or all of the audible indicators.*

## Sound processor type designations for models included in this User Manual are:

FCC ID: QZ3BAHA5, IC: 8039C-BAHA5, IC model: Baha® 5.

### Statement:

This device complies with Part 15 of the FCC Rules.

Operation is subject to the following two conditions:

- (1) this device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

**Note:** This device has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This device generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If the device does cause harmful interference to radio or television reception, which can be determined by turning the device off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the device and receiver.
- Connect the device into an outlet on a circuit different from the one in which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.
- Changes or modifications can void the user's authority to operate the device.

### Intended use

The Cochlear™ Baha® 5 Sound Processor uses bone conduction to transmit sounds to the cochlea (inner ear). It works by combining a sound processor and a small titanium implant that is placed in the skull behind the ear. The skull bone integrates with the titanium implant through a process called osseointegration. This allows sound to be conducted via the skull bone directly to the cochlea, which improves hearing performance.

### List of countries:

Not all products are available in all markets. Product availability is subject to regulatory approval in the respective markets.

The products are in compliance with the following regulatory requirements:

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