Cochlear™ Baha® Connect System

User manual part B

3	7	11	15
EN-GB	ES	FR	PT-BR

ZONE 8

English

Español

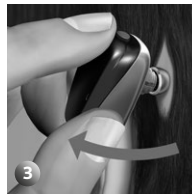
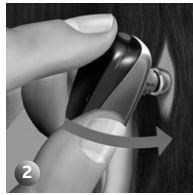
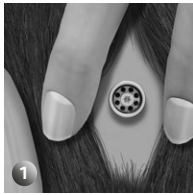
Français

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Hear now. And always


1

2

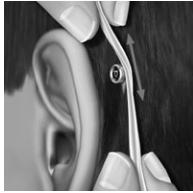3

4

Welcome

Congratulations on getting your Cochlear™ Baha® Connect System. This manual provides tips and advice on how to best use and care for your Cochlear Baha Connect System. It is a complement to the sound processor manual part A and does not replace it.

Introduction

This guide includes instructions on:

- connecting your sound processor to an abutment
- care, cleaning and maintenance
- using accessories
- important information & troubleshooting

Note on figures: The figures included on the front cover correspond to sections of information in this manual which is general for all Baha sound processors. Please reference the appropriate figure when reading. The sound processor images shown are representative only and not to scale. The appearance of the sound processor included in your kit may vary.

Connecting your sound processor

See figure **1**

Use the tilt technique to safely and comfortably connect and disconnect your sound processor, from the abutment.

- 1 Move hair from the abutment.
- 2 Hold the sound processor with the buttons positioned upwards, tilt it and gently snap it into place.
- 3 To release, place one finger under the sound processor and gently tilt until it releases.
- 4 To avoid feedback (whistling), ensure that the sound processor is not touching any other items, such as glasses or a hat.

Handling your sound processor

See figure **2**

Once you are familiar with your processor, you can change the program and the volume level while the sound processor is attached to the abutment. Make sure you support the processor from the bottom with your thumb as you press the buttons with your index finger.

Caring for your abutment

Daily and weekly tasks are required to care for your abutment properly.

Cleaning the abutment

Skin debris or other dirt in the abutment can affect the sound quality and lead to damage of your sound processor. Therefore it is very important to clean the abutment area thoroughly.

Should you experience persistent irritation, soreness or other signs of inflammation around the abutment, contact your doctor or hearing care professional.

First weeks after surgery

See figure **3**

Once the dressing is removed, maintain good daily hygiene at home to avoid redness or soreness. Wash your hair carefully and use an alcohol-free wet wipe to keep the skin clean.

Daily cleaning

After a few weeks, start cleaning the outside of the abutment daily using mild soap and plenty of warm water. An alcohol-free wet wipe should be used to remove any debris around the base of the abutment. Once clean, the area should be gently dried with a clean towel or tissue. Any hair wrapped around the abutment should be removed.

Weekly cleaning

The inside of the abutment should be cleaned on a weekly basis, using the wipes (or a soft cleaning brush) to avoid damage of the sound processor. Should you experience soreness or inflammation, or if the abutment feels loose, contact your hearing care professional.

Note: If you use a soft cleaning brush, remember to always rinse it in mild soap and warm water and let it air dry. Change the brush every three months, or if you have had an infection. If you have bilateral implants, use two separate cleaning brushes (one for each abutment), to avoid cross-contamination.

Do not use the SP Cleaning brush on the abutment due to contamination risk.

If you use the abutment cover make sure it is cleaned at least once a week. Use warm soap and water and your soft brush to clean the cover.

Long-term care

You will have regular check-ups as determined by your hearing care professional. These checkups help ensure that the processor works and fits properly and that your implant area is healthy.

Using the abutment cover

See figure **4**

The abutment cover protects the abutment when the sound processor is not in place. This prevents dirt from collecting in the abutment.

To attach the cover, gently press it onto the abutment. Lift the cover to remove.

Warnings

If you are to undergo an MRI (Magnetic Resonance Imaging) procedure, refer to the MRI Reference Card included in the document pack.

Troubleshooting

Battery function can be related to a number of problems, such as no sound, intermittent sound and crackling/buzzing. Often a new battery is the solution.

If the possible solutions in this chart do not solve your problem, please see your hearing care professional for further assistance

Problem	Cause	Possible Solution
No sound/weak sound	Weak or dead battery.	Replace battery.
	Volume too low.	Turn up volume.
Feedback (whistling)	Hat, glasses or other object in contact with sound processor.	Relocate or remove the object.
	Battery door in wrong position.	Close battery door.
Sound is distorted or intermittent	Too high volume.	Decrease volume.
	Weak battery.	Replace battery.
	Loose abutment.	Contact your hearing care professional.
Sound processor does not work	Sound processor not turned on.	Turn on sound processor.
	Battery door is not completely closed.	Gently close the battery door completely.
	Dead battery.	Replace battery.
	Battery inserted wrong way.	Make sure the battery is inserted correctly.
	Key lock is activated.	Deactivate key lock.

Additional Information

Instructions for use in the USA

Caution: Federal law (USA) restricts this device to sale or by the order of a medical practitioner.

Warning to hearing aid dispensers

A prospective Baha user must always have a medical evaluation by a licensed physician (preferably an ear specialist) and have an implant placed in the skull bone before a Baha sound processor can be dispensed. The Baha sound processor can be used with a Softband or a SoundArc which does not require an implant. However, a medical evaluation is still warranted.

Important notice for prospective users of a Baha sound processor

- A prospective Baha user must always have a thorough medical evaluation by a licensed physician (preferably a physician who specializes in diseases of the ear, otolaryngologist, otologist, or otorhinolaryngologist) and have an implant placed in the skull bone before a Baha sound processor is purchased.
- The purpose of the medical evaluation is to ensure that all medically treatable conditions that may affect hearing are identified and treated before the sound processor is purchased. Following the medical evaluation, the physician will give you a written statement that states that your hearing loss has been medically evaluated and that you may be considered a Baha candidate. The physician will refer you to an audiologist or a hearing aid dispenser, as appropriate, for an audiological evaluation.

- Federal law restricts the sale of Baha sound processors to those individuals who have obtained a medical evaluation from a licensed physician. Federal law permits a fully informed adult to sign a waiver statement declining the medical evaluation for religious or personal beliefs that preclude consultation with a physician. The exercise of such a waiver is not in your best health interest and its use is strongly discouraged.

A word of caution about batteries

Batteries can be harmful if swallowed. Be sure to always keep your batteries out of reach of small children, pets, or people of diminished capacity. In the event a battery is accidentally swallowed, seek prompt medical attention at the nearest emergency center, or call the National Button Battery Hotline collect at (202) 625 – 3333.

UKCA mark

The UKCA mark is only applicable to Baha 6 Max.

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[GB] UK Responsible Person: Cochlear Europe Ltd, 6 Dashwood Lang Road, Bourne Business Park, Addlestone, Surrey, KT15 2HJ, United Kingdom. Tel:+44 1932 26 3400

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[SG] Cochlear Limited (Singapore Branch), 238A Thomson Road #25-06, Novena Square Office Tower A, Singapore 307684, Singapore. Phone: +65 65533814

[CN] Cochlear Medical Device (Beijing) Co Ltd, Unit 2608-2617, 26th Floor, No.9 Building, No.91 Jianguo Road, Chaoyang District, Beijing 100022, P.R. China. Tel:+86 10 5909 7800

[HK] Cochlear (HK) Ltd, Room 1404-1406, 14/F, Leighton Centre, 77 Leighton Road, Causeway Bay, Hong Kong. Tel:+852 2530 5773

[IN] Cochlear Medical Device Company India PVT Ltd, Platina Bldg, Ground Floor, Plot No. C 59, G Block, BKC, Bandra East, Mumbai 400051 India. Tel:+91 22 6112 1111

[CO] Cochlear Colombia, Avenida Carrera 9 #115-06 Of. 1201 Edificio Tierra Firme, Bogota D.C., Colombia. Tel:+57 315 339 7169 / +57 315 332 5483

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www.cochlear.com

Please seek advice from your health professional about treatments for hearing loss. Outcomes may vary, and your health professional will advise you about the factors which could affect your outcome. Always follow the directions for use. Not all products are available in all countries. Please contact your local Cochlear representative for product information. In Australia, Baha bone conduction implant systems are intended for the treatment of moderate to profound hearing loss. For the Cochlear Baha System in Australia: This product is not available for purchase by the general public. For information on funding and reimbursement please contact your healthcare professional.

Cochlear, Baha, 科利耳, コクレア, 코클리어, Hear now. And always, SmartSound, the elliptical logo, and marks bearing an ® or ™ symbol, are either trademarks or registered trademarks of the Cochlear group of companies (unless otherwise noted).

© Cochlear Bone Anchored Solutions AB 2023. All rights reserved. 2023-01

2797

0086

610129-V6

93830 610129-V6

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[SE] Cochlear Bone Anchored Solutions AB, Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden. Tel:+46 31 792 44 00
[CHIREP] [CH] Cochlear AG, Peter Merian-Weg 4, 4052 Basel, Switzerland. Tel:+41 61 205 8204
[DE] Cochlear Deutschland GmbH & Co. KG, Mailänder Straße 4a, 30539 Hannover, Germany. Tel:+49 511 542 770
[SE] Cochlear Nordic AB, Konstruktionsvägen 14, 435 33 Mölnlycke, Sweden. Tel:+46 31 335 14 61
[CA] Cochlear Canada Inc, 2500-120 Adelaide Street West, Toronto, ON M5H 1T1, Canada. Tel:+1 800 483 3123