




Nucleus® Freedom™ BTE Speech Processor Quick Reference Guide

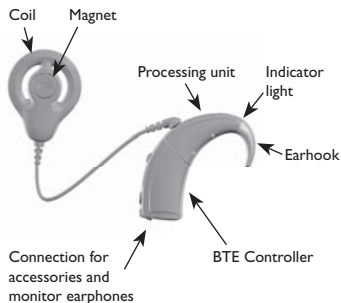

Note: There is no accessory connection with the Mini BTE Controller.

Parts of your speech processor



On and off

Press the Select button until you hear sound or see the symbols in the display.

Changing the program

Press the Select button repeatedly until you reach the program you want.

Changing microphone sensitivity

Press the Increase or Decrease buttons until you reach the level you want.

Changing batteries

Remove the battery holder and replace the batteries.



Lock and unlock

Press the Select button and the Decrease button to lock or unlock the processor.

Water

Your Freedom speech processor is protected against splashing water. Take care not to immerse it in water. If accidentally immersed in water, wipe your processor dry, remove the Controller, microphone protector and coil from the Processing Unit and remove the battery holder from the Controller. Place all parts in your dry aid kit for at least twelve hours.

If it then does not work, return it to your clinician. Cochlear cannot guarantee that they will be able to repair any water-damaged part.

Program	Description of the program and/or when to use the program
P1	
P2	
P3	
P4	

Magnet strength:

Transmitting cable length:

Nucleus is a registered trademark of Cochlear Limited. Cochlear, the elliptical logo and Freedom are trademarks of Cochlear Limited. © Cochlear Limited 2006

Cochlear Ltd (ABN 96 002 618 073) 14 Mars Road, Lane Cove NSW 2066, Australia Tel: 61 2 9428 6555


Cochlear Americas USA Tel: 1 303 790 9010

Cochlear AG Switzerland Tel: 41 61 205 0404

Cochlear Deutschland GmbH & Co. KG Germany Tel: 49 511 542 770


Printed in Australia
N30781 ISS4

Using an accessory and/or the monitor earphones

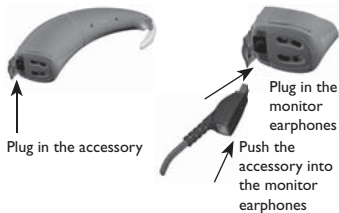

To activate the accessory, push the Increase and Decrease buttons at the same time for at least one second.

To select sound from the accessory, the in-built telecoil, or the microphone in turn, push the Increase and Decrease buttons at the same time for at least one second.

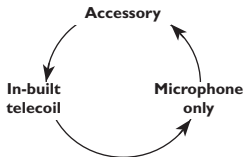

Using the in-built telecoil

Push both the Increase and Decrease buttons until 'T' shows on the display.



Changing the volume

1. Push *either* the Increase or Decrease buttons for more than one second, until you hear a beep and/or see a flashing 'V' on the display.
2. Continue briefly pushing either the Increase or Decrease buttons until you reach the volume level you want.



Note: If you do not have Sensitivity enabled, you can access Volume with a short push of either the Increase or Decrease buttons.

Resetting the speech processor

Press all three buttons (Select, Increase and Decrease) together (for at least one second) to return to the default settings for a MAP.




Changing the microphone protector



Mic lock

Attach the Mic lock as shown.

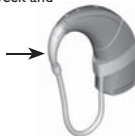

Display Key

M	Microphone
T	Telecoil
MT	Microphone and Telecoil on at the same time
S	Sensitivity (ranges from 0 to 20)
V	Volume (ranges from 0 to 9)
+L	Button controls are locked and cannot be adjusted
-L	Button controls are unlocked and can be adjusted
EA	External Accessory; flashes when accessory turned on
≡	Lines on left: processor programmed for the left ear
≡	Lines on right: processor programmed for the right ear

Help message	Meaning	Action
H1	Flat battery	Change all batteries. Note: you will stop hearing sound and only be able to turn off the speech processor.
H2	Low battery	Change all batteries.
H3	Coil error	Check: <ul style="list-style-type: none"> • coil position • cable connections • for cable damage.
H4	Audio error	Follow the steps in the <i>Troubleshooting</i> section of your User Manual to try to resolve the problem.
H5	Program corrupted	Use an alternate program when it is available until you are able to return the speech processor to your clinician. They can install a new program for you.