



A DAY with
MY SOUND PROCESSOR


Turning On

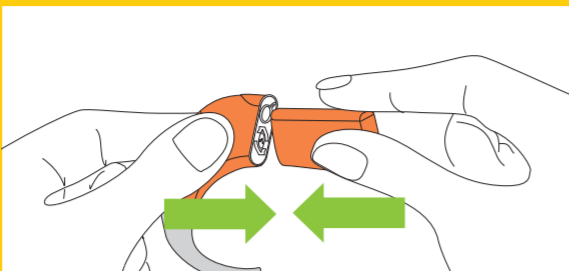


1 Hold the battery module and the processing unit so they touch.

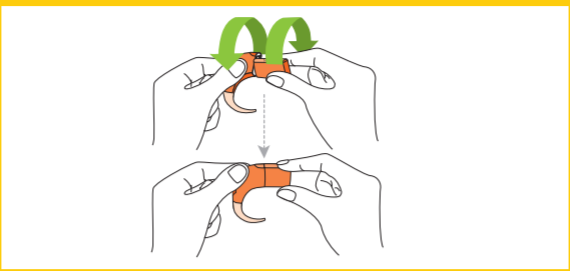

2 Twist the battery module and processing unit together to attach the two parts.

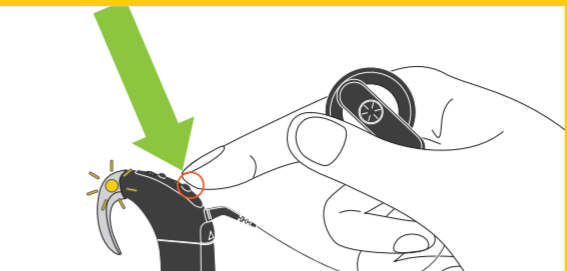


3 To turn the processor on, press the lower button for two seconds.

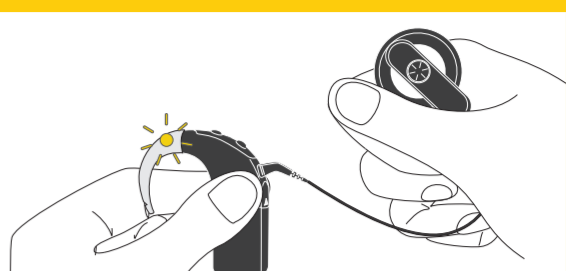

4 After the light flashes, your processor is ready to be worn.


Wearing

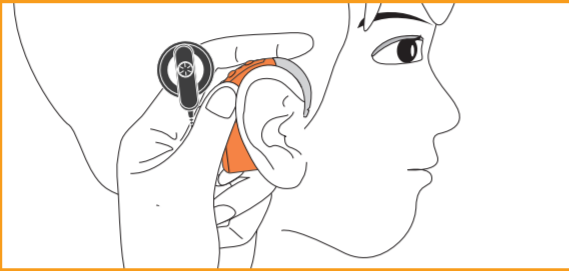


1 This is how a processor is worn correctly.



2 Place the coil on the head over the implant.

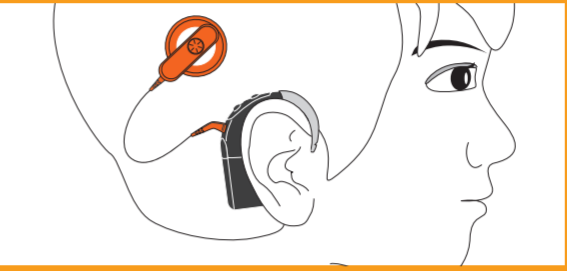

3 The coil magnet will hold the coil in place over the implant.

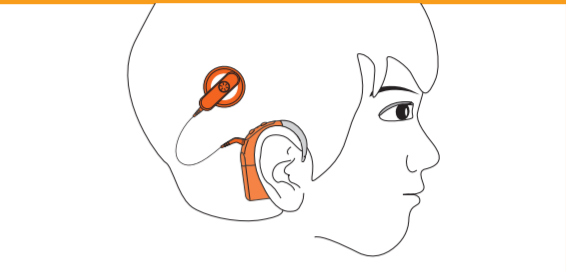


4 Take the processor and coil off the head before you go to bed.


Storing




1 Take the processor and coil off the head.

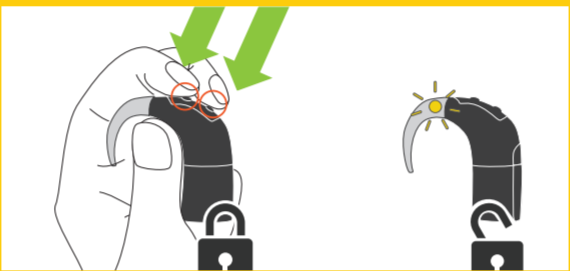

2 If the processor is locked, press both buttons to unlock.

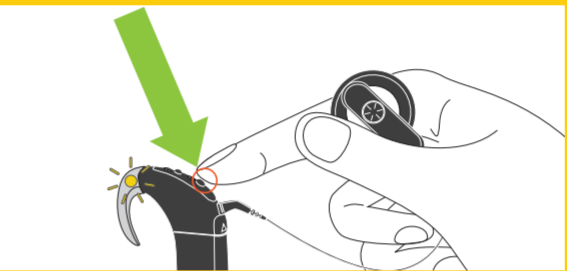

3 To turn the processor off, press the lower button for two seconds.

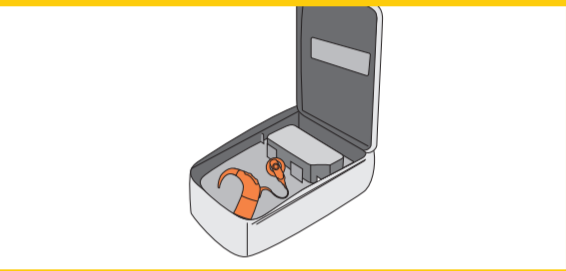

4 Store the processor in the Dry & Store unit. The battery module can remain attached if you are not charging it.

Charging




1 Twist as shown to remove the battery module from the processing unit.



2 Place the battery module on the charger.



3 Twist to connect the battery to the charger. Turn power ON.

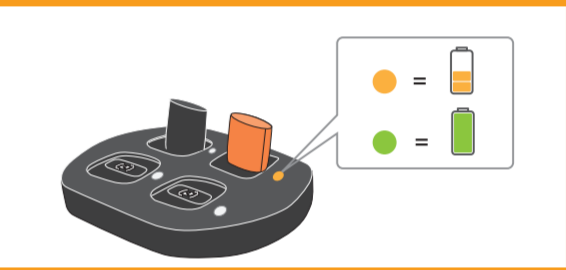

4 When the battery is fully charged, the light goes green.

Checking




Orange flashes on the processor may mean processor needs attention.



Check if the coil has fallen off, or if it is not connected properly to the implant.



OR Check if the battery is low. You may need to charge or change the battery.

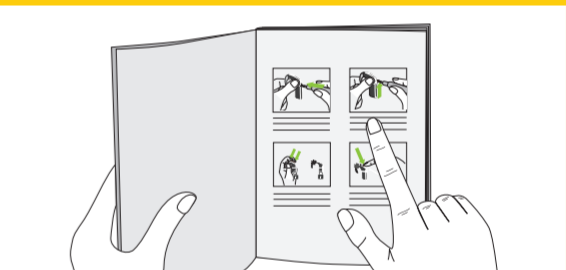

OR Problem not fixed? Please see your sound processor user guide for more information.

FIRST 3 MONTHS FROM SWITCH-ON

Note: Separate calendars are available for children and adults. Please contact your clinician for more information.

Week	MON	TUES	WED	THUR	FRI	SAT	SUN
1							
2							
3							
4							
5							
6							
7							
8							
9							
10							
11							
12							

Listening Milestones for Children*



Switch-on to 3 months

- Wears processor all day, every day.
- Reacts to sounds and voices around them.
- Recognises familiar voices and household sounds (e.g. dog barking).
- Reacts to Ling sounds 'a', 'ee', 'oo', 'm', 'sh', 's'.

Listening Milestones for Adults*



Month 1

- May recognise familiar household sounds.
- May understand familiar phrases used around the home.

Month 2

- May participate in a conversation with someone they know.
- May participate in conversations with two or three other people in a quiet environment with the aid of lip-reading.

Month 3

- May understand some television.

* Development timings are only indicative and will vary according to a number of factors.

Cochlear Ltd (ABN 96 002 618 073)
1 University Avenue, Macquarie University, NSW 2109, Australia
Tel: +61 2 9428 6555 Fax: +61 2 9428 6352
Cochlear Ltd (ABN 96 002 618 073)
14 Mars Road, Lane Cove, NSW 2066, Australia
Tel: +61 2 9428 6555 Fax: +61 2 9428 6352
Cochlear Americas
13059 E Peakview Avenue, Centennial, CO 80111, USA
Tel: +1 303 790 9010 Fax: +1 303 792 9025
Cochlear Deutschland GmbH & Co. KG
Karl-Wiechert-Allee 76A, 30625 Hannover, Germany
Tel: +49 511 542 770 Fax: +49 511 542 7770
www.cochlear.com

My to-do list:



Switch-on day:
Once healed from surgery, you will get "switched on" by your clinician.



Increase loudness:
Press the lower button to change programs as advised by your clinician.



Visit your clinician:
This is a simple way to remind you of the next clinical visit.

My clinician's details

Name: _____
Phone: _____
Address: _____